Otto Bock®

QUALITY FOR LIFE

Cloud™ Cushion

- ⓐ Instructions for Use
- ⓓ Anweisungen für die korrekte Anpassung
- ⓕ Notice de montage
- ⓓ Istruzioni per il montaggio
- ⓔ Instrucciones de ajuste
- Ⓢ Inpassningsanvisningar
- ⓑ Instruções de Colocação
- Ⓝ Instructies voor het aanpassen
- ⓐ 装着方法

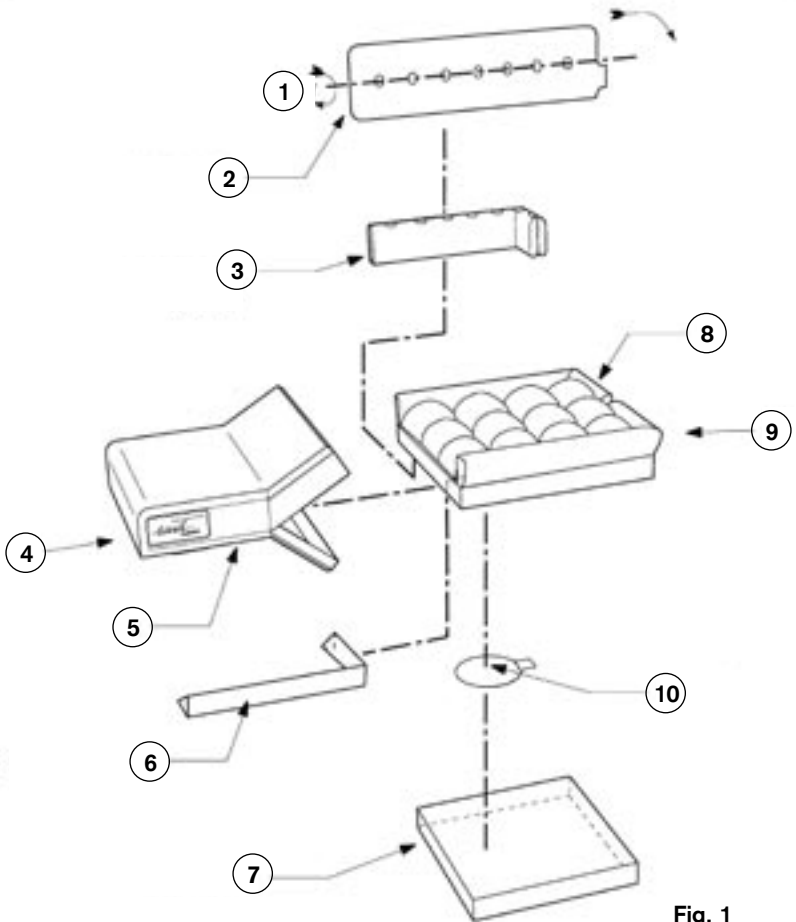
Fig. 1
Abb. 1
Afb. 1
☒1

The Cloud is to be used to improve the seated posture of the user as well as to provide pressure relief, by properly distributing existing pressure over the largest possible area. The Cloud wheelchair cushion combines superior pressure relief and maximum positioning capability to offer increased clinical benefit and comfort to the involved wheelchair user.

Declaration of Conformity

Otto Bock as manufacturer with sole responsibility declares that the Cloud™ Cushion conforms to the requirements of the 93/42 EEC Guidelines.

Safety Instructions

Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheel-chair to tip backwards, potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability. All Otto Bock cushions are designed to help reduce the incidence of pressure sores. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. Otto Bock cushions are not a substitute for good skin care including, proper diet, cleanliness, and regular pressure reliefs.

In addition, prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue break down is redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist. [In order to achieve maximum pressure relief and positioning it may be necessary to reconfigure the cells within your cushion. This should only be done after consulting with a clinician.]

Avoid sharp objects or exposure to excessive heat or open flame.

Cloud™ Cushion Assembly Instructions (Fig. 1)

1. Fold
2. Side Rail Cell
3. Side Rail Cell (folded)
4. Front
5. Cover
6. Side Foam Support
7. Foam Base
8. Cel-Pak™
9. Back
10. Floam® Cell

Application

Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. Replace worn-out padding or purchase the individual adaptable Adjustable Seat or insert a rigid plywood base in the correct size for the cushion. Avoid using overly stretched backrest upholstery. Stretched backrest upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched backrest upholstery or purchase one of Otto Bock HealthCare's

Back products in the size and configuration suitable for you. Contact an authorized supplier for a free demonstration of these products.

Care, Cleaning and Assembly

Monthly cleaning and regular maintenance may help extend the life of your cushion.

To Clean Component Parts of the Cloud Cushion:

Cover: Remove the cover from the Cel-Pak™. Turn the cover inside out, and machine wash in warm water and drip dry. No ironing required.

Attention: Do not dry clean the cover and do not wash in an industrial washing machine or tumble-dry. The cover must not be steamed or bleached.

After washing pull the cover over the Cel-Pak™ again. Make sure that the back side of the cover reaches up to the back side of the Cel-Pak™.

Note: If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.

Use only Otto Bock-designed covers for the Cloud™ cushion.

Foam Base: Remove foam base from Cel-Pak™. Wipe with warm water and soap. Wipe off with a clean cloth. Note: Avoid submersion of the foam base in water.

Cel-Pak™: Remove the cover from the Cel-Pak™. Remove the foam base, Floam™ cells, and foam rails from the Cel Pak™ and wipe cells with warm water. Dry cells off with a clean cloth. Machine wash the Cel-Pak™ in warm water and drip dry. No ironing required. Reinstall Floam™ cells and foam rails in the Cel-Pak™, then reinstall the foam base.

Re-installing Floam™ side rails: With the cell pack lying on the table upside down (colored side down):

1. Locate the "L" shaped pocket in the cell pack found on the right and left sides of the Cel-Pak™. Note: that the pocket wraps around the back edge of the cushion.
2. Fold one of the rail bladders length-wise and insert it into "L" shaped pocket in the Cel-Pak™.
3. Continue feeding the rail bladder into the Cel-Pak™ while stuffing it around the corner toward the rear of the cushion.
4. Fold the other rail bladder in half, length-wise, and insert it in the Cel-Pak™ toward the front edge of the cushion, moving it in the Cel-Pak™ as necessary to fill the remaining volume.
5. After the rail bladders are positioned in the Cel-Pak™, flip the Cel-Pak™ over (colored side up) and kneed the rail bladders to evenly distribute the Floam around the side and back edge of the cushion.

Floam Cell Adjustment: The individual Floam™ cells located inside the Cel-Pak™ can be adjusted to accommodate the user's specific anatomic shape. After removing the cover, pull out the foam base in order to access the Floam™ cells. You will find a small incision located on the under side of each Lycra pocket. You can remove the Floam™ cell located in the pocket or add additional Floam™ cells to create the necessary contouring. Remember, the different color Floam™ cells represent different volumes of Floam™.

Red = 125mg of Floam (high fill)

Black = 100mg of Floam (medium fill)

Blue = 75mg of Floam (low fill)

Daily Maintenance: To optimize the pressure relieving performance of your Cloud™ cushion, daily maintenance is required. To maintain your cushion, remove it from your wheelchair and “fluff” the cushion by folding it and unfolding it 3-4 times from different points on the cushion. When you have completed “fluffing” the cushion, the Cel-Pak™ should appear rounded and full. You can also manually fluff each Floam™ component of the cushion by removing them from the Cel-Pak™ and working the Floam™ with your hands to redistribute it throughout the cell and side rail.

Limited Warranty

Otto Bock HealthCare (Otto Bock) warrants all of its products and services, to the original purchaser, to be free from defects in materials and workmanship. This warranty applies, subject to normal wear and tear, when the products are used as intended, without unapproved modifications, following all Otto Bock instructions and requirements; and when they are fitted by or under the direct supervision of certified/licensed practitioners who meet all Otto Bock product-specific training requirements for a given product.

The duration of our Limited Warranties varies by product and service, but all are effective from the date of delivery to the patient. When indicated, the duration of the Limited Warranty for some components may be extended when incorporated into a finished device in which all of the components are manufactured by Otto Bock and installed in conformance with all Otto Bock recommendations. Otto Bock’s sole obligation under this Limited Warranty shall be to repair, replace, or refabricate at no charge, or to refund the cost of the system or component thereof to the original purchaser, at Otto Bock’s sole discretion. The specific terms and conditions of the limited warranty for the following product(s) are as follows:

Cloud Cushion

• Warranty Duration – Parts	24 months
• Warranty Duration – Fabrication	N/A
• Extended warranty available	No
• Manufacturer training required	No
• Service required	No

This limited warranty does not cover labor, freight charges, or damage due to accidents, neglect, misuse or operation beyond capacity, parts damaged by improper installation, substitution of parts not approved by Otto Bock, any alteration or repair by others that, in Otto Bock’s judgment, materially or adversely affects the product or part.

THE EXPRESS WARRANTIES SET FORTH HEREINABOVE ARE IN LIEU OF ALL OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND ALL SUCH OTHER WARRANTIES ARE HEREBY DISCLAIMED AND EXCLUDED BY OTTO BOCK. IN NO EVENT SHALL OTTO BOCK’S LIABILITY OF ANY KIND INCLUDE ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF OTTO BOCK SHALL HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH POTENTIAL LOSS OR DAMAGE.

Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

Please refer to the Otto Bock website or product/service price list for additional warranty information or contact your local Otto Bock Sales office.

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Otto Bock HealthCare GmbH