
**Otto Bock®**

QUALITY FOR LIFE


# Otto Bock Back Collection

- ⓐ Instructions for Use
- ⓓ Anweisungen für die korrekte Anpassung
- ⓕ Notice de montage
- ⓖ Istruzioni per il montaggio
- ⓔ Instrucciones de ajuste
- Ⓢ Inpassningsanvisningar
- ⓖ Instruções de Colocação
- Ⓝ Instructies voor het aanpassen
- ⓐ 装着方法


1.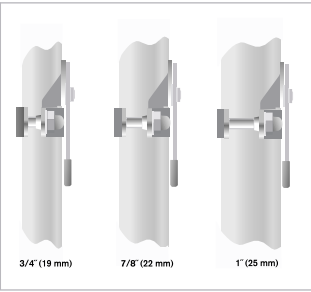


2.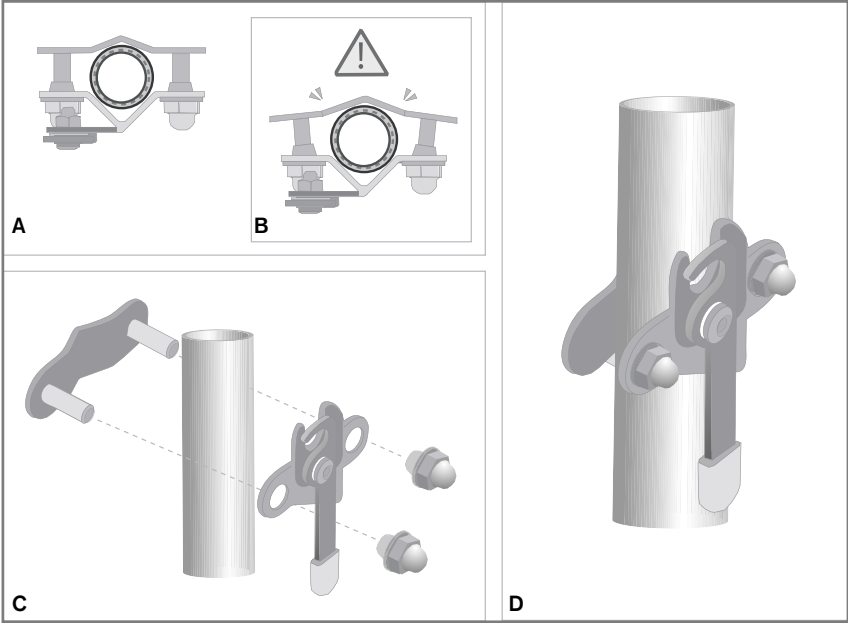


3.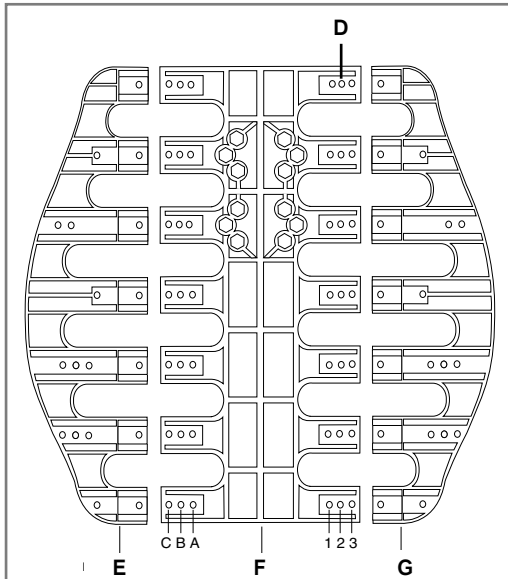


4.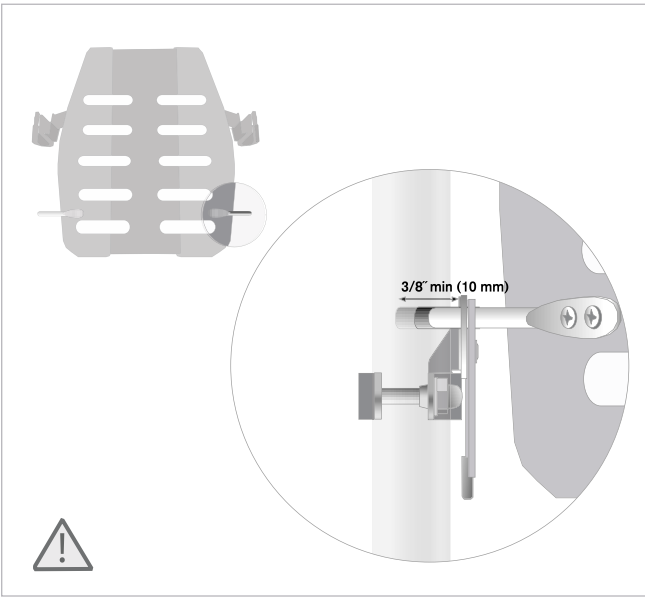


5.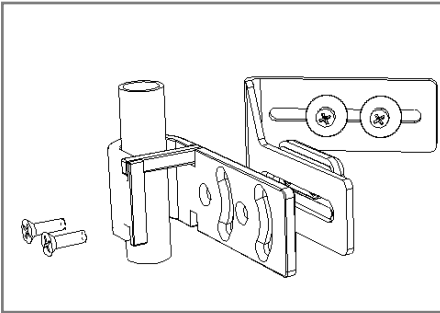


6.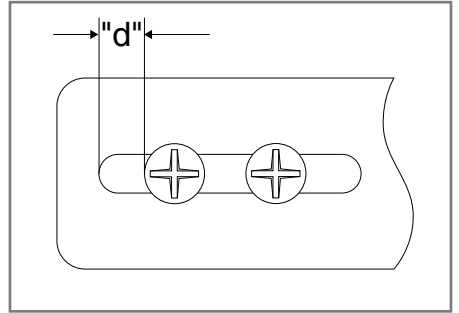


7A.

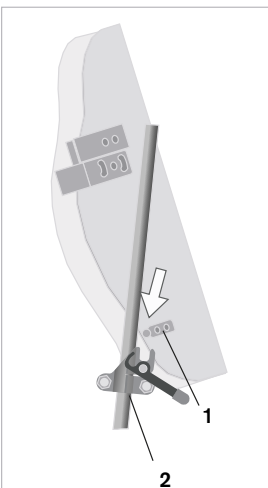
7B.

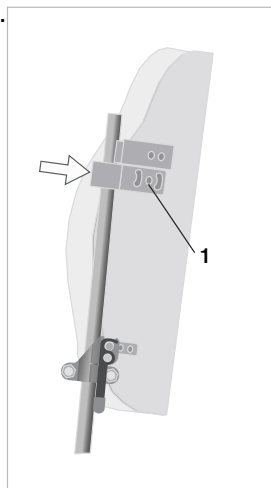
7C.

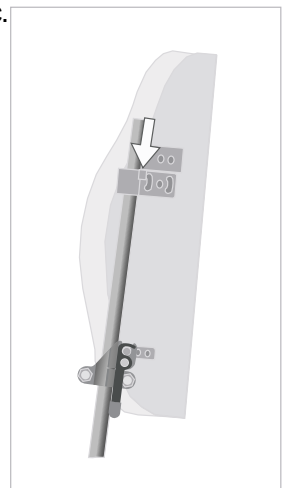
8.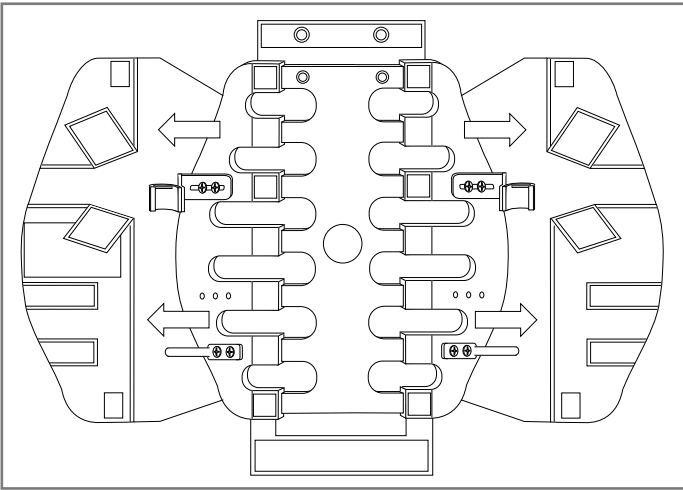


9.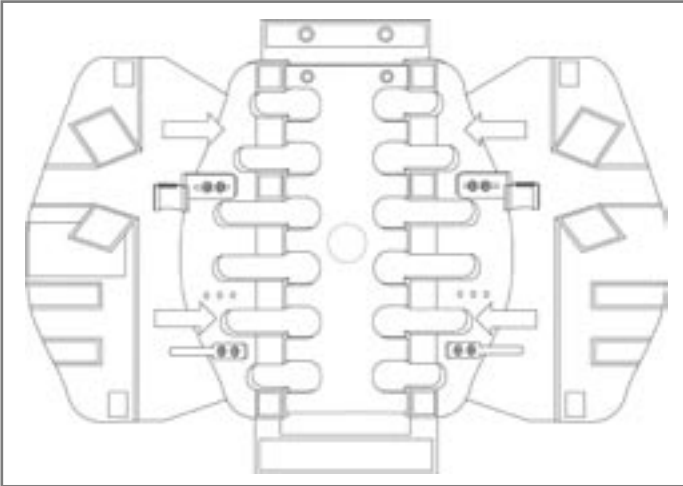


10.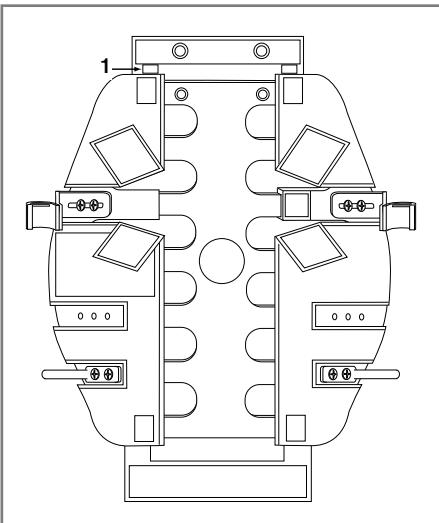


11.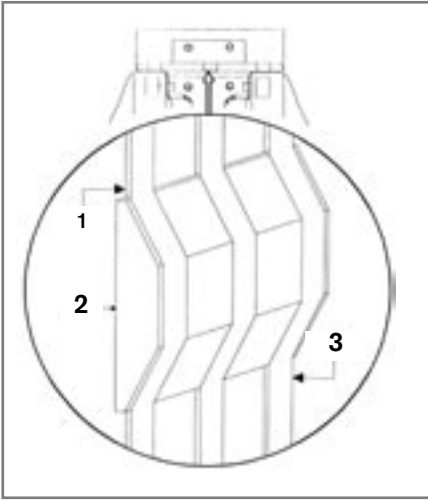

12.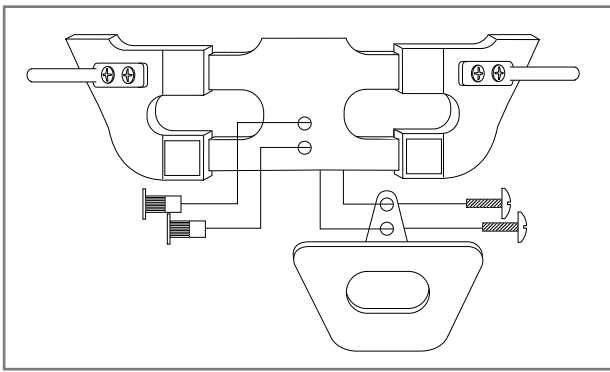

13.

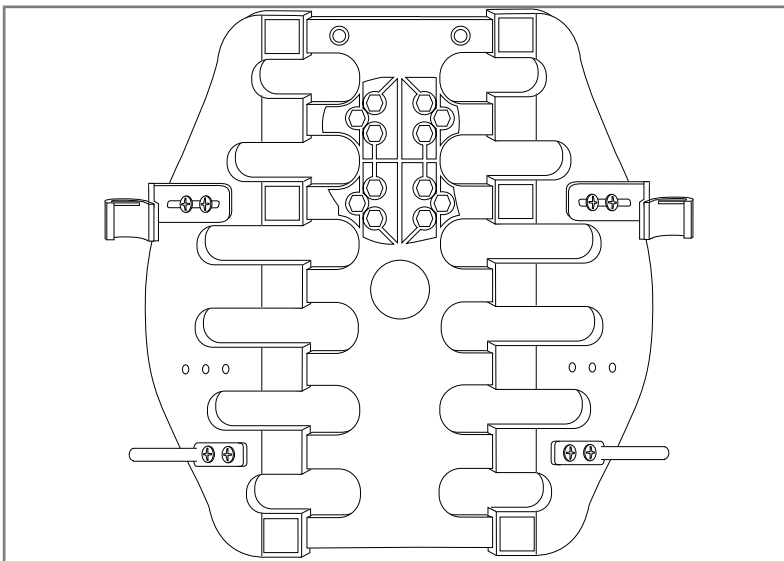**Otto Bock Back 1****Otto Bock Back 2****Otto Bock Back 3****Otto Bock Back 4****Otto Bock Breathable™ Back**

By Otto Bock HealthCare

**Intended Use and Descriptions**

Otto Bock's family of wheelchair back cushions are used to improve the seated posture of the user, as well as to provide pressure relief, by properly supporting the trunk region and distributing existing pressure over the largest possible area.

**Application**

Improved thoracic lateral support if the user's sitting posture requires additional support.

Support for an orthopedic deformity such as a lordosis, can be inserted in a compartment between the solid base and the pad.

Using the integrated FLOAM® pads for Back 2, Back 3 and Back 4 in the vertebral region can reduce risk of decubitus ulcers.

The Otto Bock warranty applies only when the product is used according to the specified conditions and for the intended purposes, following all manufacturers' recommendations (see 2-Year Limited Warranty section for more details).

**INSTRUCTION MANUAL****BEFORE YOU BEGIN:****Tools Needed:**

Phillips head screwdriver.

Soft-headed hammer: optional width adjustment.

Drill with 1/4" (6 mm) bit: for use in attaching optional headrest.

**Preparing the Wheelchair:**

Remove the upholstery from the back of the wheelchair's push canes per manufacturer's instructions.

**Definitions as Used in this Manual:**

1. B1 (Back 1)
2. B2 (Back 2)
3. B3 (Back 3)
4. B4 (Back 4)

**Installing the Otto Bock Back Cushion to the Wheelchair****A. Connecting the Rigid Back Bracket to the Wheelchair Push Canes**

1. Determine the diameter of the wheelchair push canes.
2. The rigid back bracket will accommodate 3/4", 7/8", and 1" (19-20 mm) push canes (Figure 1). Hold the Back in the location it will be installed on the wheelchair and place a mark on both push canes to indicate the location for the bracket tape. (This location is about 1/4" (6 mm) below the rigid back bars).
3. Remove the rigid back bracket screw and attach the rigid back bracket around the wheelchair push canes as shown in Figure 2.
4. Make sure that the left and right rigid back brackets are the same height on the wheelchair push canes before tightening.
5. Re-insert the rigid back bracket screw and tighten as shown in Figure 2A, 2C, 2D.

**NOTE:** The rigid back bar notch can be positioned toward the posterior of the wheelchair as shown. The rigid back bar notch can also be in the anterior position, although seat depth will be affected.

**NOTE:** Avoid over-tightening the screws around the wheelchair push canes (See Figure 2B)

## B. Adjusting the Back Width

A unique feature of the Back is its range of width adjustability. It is designed first to custom fit the user, and second, to fit onto the wheelchair. This is accomplished by having two sets of width adjustments.

When the proper width of the Back has been adjusted to the specific user, a second set of adjustments within the attachment hardware allows the Back to properly fit the wheelchair. The Back is offered in two standard sizes:

	<b>Model Number</b>	<b>Description</b>
<b>B1</b>	476D10=RA15-17	Standard model - accommodates w/c back widths between 15" (38 cm) and 17" (43 cm); is factory preset at 16" (41 cm)
	476D10=RA18-20	Large model - accommodates w/c back widths between 18" (46 cm) and 20" (51 cm); is factory preset at 18" (46 cm)
<b>B2</b>	476D20=AR15-17	Standard model - accommodates w/c back widths between 15" (38 cm) and 17" (43 cm); is factory preset at 16" (41 cm)
	476D20=AR18-20	Large model - accommodates w/c back widths between 18" (46 cm) and 20" (51 cm); is factory preset at 18" (46 cm)
<b>B3</b>	476D30=ARD15-17	Standard model - accommodates w/c back width between 15" (38 cm) and 17" (43 cm); is factory preset at 16" (41 cm)
	476D30=ARD18-20	Large model - accommodates w/c back widths between 18" (46 cm) and 20" (51 cm); is factory preset at 18" (46 cm)
	476D30=ARTD15-17	Standard model - accommodates w/c back widths between 15" (38 cm) and 17" (43 cm); is factory preset at 16" (41 cm); includes tear drop laterals
	476D30=ARTD18-20	Large model - accommodates w/c back widths between 18" (46 cm) and 20" (51 cm); is factory preset at 18" (46 cm); includes tear drop laterals
<b>B4</b>	476D40=AR4D15-17	Standard model - accommodates w/c back widths between 15" (38 cm) and 17" (43 cm); is factory preset at 16" (41 cm)
	476D40=AR4D18-20	Large model - accommodates w/c back widths between 18" (46 cm) and 20" (51 cm); is factory preset at 18" (46 cm)
	476D40=AR4TD15-17	Standard model - accommodates w/c back widths between 15" (38 cm) and 17" (43 cm); is factory preset at 16" (41 cm); includes tear drop laterals
	476D40=AR4TD18-20	Large model - accommodates w/c back widths between 18" (46 cm) and 20" (51 cm); is factory preset at 18" (46 cm); includes tear drop laterals

### Otto Bock Breathable Back

476D11=RB15-17	Standard Model - accommodates wheelchairs ranging from 15" (38 cm) to 17" (43 cm) wide.
476D11=RB18-20	Standard Model - accommodates wheelchairs ranging from 18" (46 cm) to 20" (51 cm) wide.

*Table 1. Model Numbers*

**Note:** Both sizes (standard and large) allow micro adjustments of 1/2" (12 mm) increments within the range of adjustability.

*Tips before you begin (Width adjustments)*

1. The mounting hardware offers about 2" (50 mm) of total adjustability. This means that the push canes of the wheelchair, as a general rule, need to be within 2" (50 mm) of the width chosen for the user.

The foam pad (B1), FLOAM® Pad (B2 & B3) or Cel-Pak (B4) are designed to expand and to accommodate all widths within the range of adjustability for each back.

**B.1 Adjusting the Back Width to Fit the User**

1. Measure the width of the user's back. This measurement represents a starting width setting. If this width differs from the factory settings, follow the adjustment Steps 2-8 below, otherwise skip ahead to Step 8.
2. Remove the upholstery by detaching all points of hook and loop fasteners and unsnapping the two snaps on the top rear side of the Back. Refer to section "D. Removal of the Upholstery".
3. With a Phillips screwdriver, remove the "rigid back wing-to-spine" attachment screws. Figure 3, Pos. D.

**NOTE:** The rigid back wing will not slide in or out of the spine until all of the attachment screws have been removed.

4. For exact width specifications, refer to Figure 3 and Table 2 for proper hole configuration.

**Figure 3 Arrangement:**

- Pos. D Attachment Screws
- Pos. E Right Wing
- Pos. F Spine
- Pos. G Left

*Table 2. Proper hole alignment.*

For Desired Width	Use Holes		For Desired Width
	Left wing	Right wing	
15" (38 cm)	A	1	18" (46 cm)
15 1/2" (39 cm)	A	2	18 1/2" (47 cm)
	B	1	
16" (41 cm)	B	2	19" (48 cm)
16 1/2" (42 cm)	B	3	19 1/2" (50 cm)
	C	2	
17" (43 cm)	C	3	20" (51 cm)

5. Once the proper hole pattern is chosen, insert the screws in the top and bottom holes only, so that the width can be checked for proper patient fit.
6. Before you proceed check the Back width as follows:
  - a. Re-attach the upholstery as described in section "E. Installation of the Upholstery".
  - b. Attach the Back to the wheelchair push canes. If the rigid back hooks do not fit the width of the wheelchair and need adjustment, make adjustment as described in section B.2.
  - c. With the new back in place, have the user sit in the wheelchair and check for proper fit.

- d. When the fit is proper, remove the foam pad upholstery and insert the remaining attachment screws. Tighten securely. If the fit is not proper, it may be necessary to repeat steps 2-6d.
7. Replace the upholstery as described in section "E. Installation of the Upholstery."
8. The Back is ready for final installation.

### ***B.2 Adjusting the Back to Properly Fit the Wheelchair***

With the Back properly fitted for the user, it is time to make sure that it fits the wheelchair.

1. With the Back installed the rigid back bar should be extended to the outer edge of the rigid back bracket notch as shown in Figure 3.
2. The Rigid Back Hook Assembly can be width adjusted as shown in Figure 4.
  - a. If the wheelchair push canes are less than 1" (25 mm) diameter, attach one layer of adapter tape for 7/8" (22 mm) tubes or two layers of tape for 3/4" (19 mm) tubes (see Figure 1).
  - b. Loosen the two rigid back offset wedges to wing adjustment screws and slide the Rigid Back Hook Assembly in or out so that the rigid back hook matches the width of the wheelchair as shown in Figure 5.

**NOTE:** Use a Phillips screwdriver for the 10/32 x 5/8 Phillips screws.

- c. Once both widths are matched, tighten the adjustment screws.

**NOTE:** It is important that both the left and right rigid back hook assemblies are moved the same distance as shown in Figure 5. If not, the Back will sit crooked in the wheelchair. The definition of left and right side, is called out, as the user sits in the wheelchair. The user's left side is the Back left side and the wheelchair's left side.

### ***B.3 Attaching the Back to the Wheelchair***

1. Insert the rigid back bars (Figure 7A, Pos. 1) into the receiving lip of the back bracket (Figure 7A, Pos. 2). Pull the closing latch of the bracket securely over the back bars on each side of the back. (Figure 7B)
2. With the rigid back hook latches (Figure 7B) lifted up, place the rigid back hooks (Figure 7B, Pos. 1) against the wheelchair push canes.
3. Lock the rigid back hook latches in place (down position) as shown in Figure 7C to prevent the Back from rotating forward.

## **C. Adjusting the Recline or Incline**

The Back is factory preset at a 90° back angle. When installed, the posterior surface will be parallel to the push canes. It is possible to adjust the back angle from approximately 80° to just over 100°.

1. Each rigid back hook assembly is held together with two 10/32 x 5/8 flathead screws and nut sliders (Figure 5).
2. To change the angle, remove both screws and nuts from each side of the rigid back hook assembly and adjust as needed.

**NOTE:** Both screws must be used to ensure proper use.

3. Once the proper setting has been determined, insert the two screws into the proper holes and firmly re-tighten.

## **D. Removal of the Upholstery**

1. Unsnap the tabs and remove the foam pad (B1), FLOAM® Pad (B2 & B3) or Cel-Pak (B4) completely from the Back as shown in Figure 8.
2. Separate all hook and loop fasteners and remove the wing covers (B2, B3 and B4 only) from the wings as shown in Figure 8.

## E. Installation of the Upholstery

1. Slip the wing sections over the appropriate wings and re-attach all fasteners as shown as shown in Figure 8 (B2, B3, B4 only).
2. Align the top snaps with the snaps on The Back of the center spine section, and snap into place (B1, B2, B3 and B4). Fold the expandable sections of the wings under the FLOAM® Pad or Cel-Pak and secure the bottom (B2, B3, B4 only).

## OPTIONAL EQUIPMENT

### F. Adding Lumbar or Sacral Support (B2 and B3 only) Otto Bock Pelvic Positioning/Support Kit

**Caution:** The selection, positioning and installation of these, and any other, seated positioning products should be done by a qualified therapist, Rehabilitation Technology Supplier or other trained seating specialist.

If determined by the seating specialist that the user of the Back requires additional pelvic positioning or support, the lumbar roll or sacral wedge by Otto Bock HealthCare may be added to the basic back configuration.

The lumbar roll is used primarily to increase the amount of lumbar lordosis. To avoid potential skin breakdown and discomfort to the user, the lumbar roll should never be used with a fixed or non-reducible lumbar flexion deformity.

The sacral wedge, with the wider edge toward the top and the flat surface toward the back of the chair, is designed to apply anteriorly directed forces at the level of the P.S.I.S. (posterior superior iliac spine). Its primary function is to position a flexible pelvis/lumbar spine in a slightly anteriorly tilted position. This position helps maintain the natural complementary curves of the spine.

The sacral wedge may also be used to support a fixed or non-reducible, posteriorly tilted pelvis. In this application, the sacral wedge should be positioned with the wider edge toward the bottom and the flat surface toward the back of the chair. This position supports the pelvis to help prevent additional posterior pelvic tilt.

**Please Note:** The lumbar roll and sacral wedge will be most effective when used with a well-positioned and properly adjusted pelvic positioning support, i.e., lap belt.

1. A compartment within the Back upholstery is accessible when the Back is in place, and the user is in the wheelchair. This compartment allows a sacral or lumbar support to be inserted into the Back to customize the shape of the Back.
2. There are three strips of hook and loop fasteners, running from the top to the bottom of this compartment as shown in Figure 10, Pos. 1.
3. These fasteners will allow the placement of the Otto Bock Lumbar Roll or Sacral Wedge (or various sizes or shapes of foam or FLOAM® bladders) to provide a wide range of adjustments.
4. To customize the shape of the Back, obtain the desired shape of foam or bladder, or order the Otto Bock Pelvic Positioning/Support Kit from Otto Bock Health Care. Insert the lumbar roll or sacral support into the compartment at the desired location and press the hook side (Figure 11, Pos. 1) to the loop side (Figure 11, Pos. 3) around the insert (Figure 11, Pos. 2) to hold in place.
5. Have the user lean back. The seating specialist should verify the position and function of the lumbar roll or sacral support.
6. Make adjustments as needed.
7. Recheck fit and function.

8. Snap the outside compartment.
9. The Back is ready for use.

### **G. The Tail Extension (for all backs)**

If the Back is attached to the wheelchair in a high position on the wheelchair push canes, it may be too high to prevent the seat cushion from sliding out the back of the wheelchair. An optional lower tail extension is available to eliminate this problem.

**NOTE:** The Otto Bock Tail Extension must be ordered separately from Otto Bock HealthCare using the following article number: 476D20=TAIL-EXT.

1. Remove the upholstery as described in section "D. Removal of the Upholstery".
2. Slide the tail into place, aligning the attachment holes.

**Note:** Upholstery should face the back of the user and the plastic tail should attach on the outside of the Back.

3. Insert the (2) flanged riv-nuts from the inside of the Back spine as shown in Figure 12, and insert the tail to spine screw. Tighten securely.
4. Replace the upholstery as described in section "E. Installation of the Upholstery".

### **H. Headrest Attachment (for all backs)**

The Back is designed to accommodate a wide variety of standard headrest attachments, including all Otto Bock headrest styles using the hardware supplied with the 430F2, 430F5, 430F6 and 430F7 head support fixtures. Choose one of the two sets of "skinned over" holes indicated in Figure 13. Carefully follow the instructions for headrest bracket installation below to assure safety and proper function.

1. Remove the upholstery as described in section "D. Removal of the Upholstery".
2. Align the headrest attachment brackets with the series of skinned over holes on the inward side of the spine.
3. Select the required number of holes and note which will need to be drilled.
4. Using a 1/4" (6 mm) bit, drill out the skinned over holes on the spine as shown in Figure 13.
5. Attach the mounting hardware and tighten all screws snugly. Attach the headrest per manufacturer's instruction.
6. Adjust headrest as needed.
7. Replace the upholstery as described in section "E. Installation of the Upholstery".

## **Care Guide**

### ***Care of the Foam Pad (B1 only)***

The B1 contains an open cell foam pad that must be removed prior to washing. This pad should NOT be washed. The fabric cover of the B1, however, may be machine washed on a knit/delicate cycle. DO NOT BLEACH. Drip dry.

### ***Care of the FLOAM® Pad (B2 & B3 only)***

The FLOAM® Pad has channels that prevent the FLOAM® from shifting, but will allow the FLOAM® to flow, distributing pressure. Avoid puncturing the FLOAM® Pad with sharp items.

The FLOAM® material maintains its properties in a variety of climates. Care should be taken, however, to allow the FLOAM® Pad to return to room temperature after prolonged exposure to extremes of cold or heat.

The FLOAM® Pad may be machine washed on a knit/delicate cycle. DO NOT BLEACH. Drip dry. Each side or wing section contains an open cell foam pad that should NOT be machine washed. Remove these prior to washing.

### **Care of the Cel-Pak (B4 only)**

The Cel-Pak contains several FLOAM® cells. These must be removed prior to washing. The cells are color-coded, on the Cel-Pak. The foam in the Cel-Pak must also be removed prior to washing. Re-assemble by placing the color-coded cells in the corresponding color-coded pocket i.e. place red cells in the red pockets etc., unless prescribed otherwise prescribed by a trained seating specialist. Hand wash FLOAM® cells in warm, soapy water. Rinse and pat dry.

### **Care of the Composite Shell (all backs)**

The composite material may be cleaned using a mild soap and water. The shell should be completely dry before replacing the upholstery and reinstalling it into the chair.

Should the Composite Shell get scratched or nicked, it may be restored to its original condition by smoothing the area with a fine grade sandpaper. With a soft cloth rub a small amount of baby oil over the sanded area. This will renew the original surface appearance.

### **Washing instructions for the Otto Bock Breathable† Back:**

**Breathable Cover:** machine wash warm, tumble dry low heat.

Do not bleach. Do not iron.

**Breathable foam cushion:** hand wash in warm water with a mild detergent (do not use ammonia based detergents). Air dry.

Do not bleach.

**Note:** The fabric of the Otto Bock Breathable Back may snag or dent when it comes in contact with sharp or hooked objects (such as hook and loop fasteners or jewelry). Such cosmetic changes will not affect the breathable quality of the fabric.

### **I. Limited Warranty**

Otto Bock HealthCare (Otto Bock) warrants all of its products and fabrication services, to the original purchaser, to be free from defects in materials and workmanship. This warranty applies, subject to normal wear and tear, when the products are used as intended, without unapproved modifications, following all Otto Bock instructions and requirements; and when they are fitted by or under the direct supervision of certified/licensed practitioners who meet all Otto Bock product-specific training requirements for a given product.

The duration of our Limited Warranties varies by product and service, but all are effective from the date of delivery to the patient. When indicated, the duration of the Limited Warranty for some components may be extended when incorporated into a finished device in which all of the components are manufactured by Otto Bock and installed in conformance with all Otto Bock recommendations. Otto Bock's sole obligation under this Limited Warranty shall be to repair, replace, or refabricate at no charge, or to refund the cost of the system or component thereof to the original purchaser, at Otto Bock's sole discretion. The specific terms and conditions of the limited warranty for the following product(s) are as follows:

#### **Otto Bock Back Collection**

• Warranty Duration – Parts	24 months
• Warranty Duration – Fabrication	N/A
• Extended warranty available	No
• Manufacturer training required	No
• Service required	No

This limited warranty does not cover test sockets, or damage due to accidents, neglect, misuse or operation beyond capacity, parts damaged by improper installation, substitution of parts not approved by Otto Bock, any alteration or repair by others that, in Otto Bock's judgment, materially or adversely affects the product or part.

THE EXPRESS WARRANTIES SET FORTH HEREINABOVE ARE IN LIEU OF ALL OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND ALL SUCH OTHER WARRANTIES ARE HEREBY DISCLAIMED AND EXCLUDED BY OTTO BOCK. IN NO EVENT SHALL OTTO BOCK'S LIABILITY OF ANY KIND INCLUDE ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF OTTO BOCK SHALL HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH POTENTIAL LOSS OR DAMAGE.

Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

Please refer to the Otto Bock website or product/service price list for additional warranty information or contact your local Otto Bock Sales office.

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Otto Bock HealthCare GmbH