3B1 / 3B1=ST

Genium – Bionic Prosthetic System

DE	Gebrauchsanweisung.....	2
EN	Instructions for Use.....	54

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Service-Telefon

Deutschland: (+49) 55 27-848 13 33
Österreich: (+43) 1-5 23 37 86 430

<p>Adapter</p>	 <p>4R104=60 4R104=75</p> <p>4R57 4R57=ST</p> <p>4R41 4R43 4R89</p> <p>4R111=N 4R111 4R116</p> <p>4R119</p> <p>4R40</p> <p>4R118</p>	 <p>757L16-*</p> <p>4E60-*</p>	<p>Charger/ Power supply unit</p>
<p>Prosthetic knee joint/Remote control</p>	 <p>3B1/3B1=ST</p> <p>4X350</p>	 <p>4X258</p>	<p>Installation ring and tool for inductive charger</p>
<p>AXON Tube adapter</p>	 <p>2R20</p> <p>2R21 (with Torsion)</p>	 <p>4X259</p>	<p>Installation ring and tool for inductive charger</p>
<p>Prosthetic foot</p>	 <p>1C40</p> <p>1E56</p> <p>1M10</p> <p>1C30</p> <p>1D35</p>	 <p>X-Soft 4X1=*</p> <p>BionicLink PC 60X5</p>	<p>BionicLink PC / Configuration software</p>

INFORMATION

These instructions for use were last updated on October 21th 2010.

Table of Contents

1 Intended Use	57
1.1 Medical Purpose.....	57
1.2 Field of Application	57
1.3 Conditions of Use	57
1.4 Qualifications of the Prosthetist.....	58
1.4.1 Qualifications of the Therapist and Nursing Staff	58
1.5 Functions.....	58
1.6 Safety Instructions	58
1.7 Scope of Delivery.....	58
1.8 Recommended accessories	58
1.9 General Safety Instructions.....	58
1.10 Patient Information.....	60
2 Installation of X-Soft and connection to the Bluetooth® environment	62
2.1 Minimum hardware requirements for PC with 32-bit operating systems.....	62
2.2 Minimum PC hardware requirements for PCs with 64-bit operating systems	62
2.3 Recommended PC hardware specifications for 32-bit and 64-bit operating systems.....	62
2.4 Software requirements.....	62
2.4.1 Supported operating systems	62
2.4.2 Additional required software components	63
2.5 Installing	63
2.6 Uninstalling.....	65
3 Alignment and adjustment of the prosthesis.....	65
3.1 Charging the 3B1/3B1=ST knee joint	65
3.2 Connecting the AXON Tube Adapter	67
3.3 Starting the Program	68
3.4 Closing the Program.....	69
3.5 Establish connection between the 3B1/3B1=ST knee joint and X-Soft	70
3.6 Entering the basic data in X-Soft	71
3.7 Entry of the patient's data into X-Soft.....	73
4 Torque values of the screw connections	73
5 Basic prosthetic limb alignment with X-Soft	74
5.1 Static alignment optimisation with X-Soft	77
5.1.1 Conditions for measurement validity.....	78
5.1.2 Entering measurements	78
5.2 Configuring the prosthesis control via X-Soft	80
5.2.1 Adjusting stance phase control	80
5.2.2 Adjusting the swing phase control.....	81
5.2.3 Functions in basic mode	82
5.2.4 Selecting MyModes	85
5.2.5 Setting the resistance in safety mode.....	86

5.2.6	Setting the pitch and volume of the audible warning and information signals.....	87
5.2.7	Data overview after configuring the prosthetic controls	88
6	MyMode Motion Switching	89
6.1	Switching to MyModes without a remote control	89
7	Remote control.....	90
7.1	Introduction.....	90
7.2	Control elements of the remote control.....	91
7.3	Initial connection between the remote control and the 3B1/3B1=ST knee joint (Pairing).....	91
7.4	Status bar	91
7.5	MyMode switching via remote control	92
7.6	Changing prosthesis settings.....	92
7.7	Deep sleep mode.....	93
7.8	Querying the prosthesis status.....	94
7.9	Remote control adjustments.....	95
7.10	Managing prostheses	95
7.11	Connecting to the prosthesis during normal operation.....	96
7.12	Charging the remote control	96
8	Completing the fitting.....	97
9	Attaching the optional cosmetic covers.....	97
10	Important information for users.....	98
10.1	Safety messages and safety modes.....	98
10.2	Charging status	99
10.3	Storage and bleeding	99
10.4	Service interval	99
10.5	Guarantee	99
10.6	Technical Information	99
10.7	Symbols on the knee joint	100
10.8	Symbols on the remote control.....	100
10.9	Symbols on the inductive charger	100
11	Definition of terms.....	101
12	Legal notes, liability and CE conformity for X-Soft.....	103
13	Liability.....	104
14	CE Conformity	105

Explanation of symbols in the instruction manual

⚠ WARNING Warnings regarding possible risks of severe accident or injury.

⚠ CAUTION Warnings regarding possible risks of accident or injury.

INFORMATION Additional information on the fitting/use.

1 Intended Use

INFORMATION

Read these instructions carefully before using the 3B1/3B1=ST knee joint! Pay particular attention to the safety instructions!

The patient must be taught how to handle, care for and operate his/her prosthesis properly. See the sections below: 1.3 Conditions of Use, 1.5 Functions, 1.6 Safety Instructions, 3.1 Charging the 3B1/3B1=ST Knee Joint, 6 MyMode Motion Switching, 7 Remote Control, 10 Important information for users, 13 Liability

1.1 Medical Purpose

The 3B1/3B1=ST knee joint is to be used **exclusively** for exoprosthetic fitting of the lower limbs.

1.2 Field of Application

The patient must fulfil the following requirements:

- The patient must fulfil the physical and mental requirements for the perception of audible signals and/or mechanical vibrations.
- The skin on the residual limb must be completely healed.
- The knee fulcrum center ground measurement must be at least 318 mm (if 2R20 and 1E57 are used). The minimum distal system height (distance between the knee axis and the centre point of the lower pyramid-shaped receptacle at the AXON Tube Adapter) of the 3B1/3B1=ST knee joint with a AXON Tube Adapter must be 310 mm if 2R20 is used and 344 mm if 2R21 is used.

The 3B1/3B1=ST knee joint can only be used for unilateral transfemoral amputees and patients with knee disarticulation.


INFORMATION

Advise your patients of the information in this section.

Application according to the **Otto Bock MOBIS® Mobility System**:

Recommended for **mobility grades 2, 3 and 4** (restricted outdoor walkers, unrestricted outdoor walkers and unrestricted outdoor walkers with especially high requirements).

Approved for **body weights of up to 150 kg** with 2R20, or up to **125 kg** with 2R21.

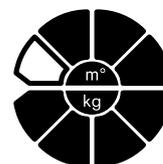1.3 Conditions of Use

INFORMATION

Advise your patients of the information in this section.

The 3B1/3B1=ST knee joint was designed for everyday activities, but not for extreme sports, such as free-climbing, skydiving, paragliding etc. See Section 10.5 Technical Information for the permitted ambient conditions. The 3B1/3B1=ST knee joint is only intended for fitting to amputees or dysmelia sufferers. Use of the product on other patients is not approved by the manufacturer.

1.4 Qualifications of the Prosthetist

Only prosthetists who have been authorised by Otto Bock after completing a corresponding training course may fit patients with the 3B1/3B1=ST knee joint.

1.4.1 Qualifications of the Therapist and Nursing Staff

The therapists or nursing staff must be trained in handling the 3B1/3B1=ST knee joints (and accessories) by the authorised prosthetist.

1.5 Functions

The 3B1/3B1=ST knee joint is an electronic knee joint system with hydraulic stance and swing phase control. It is controlled by a microprocessor-based hydraulic system that dynamically adapts the system in real time. The control mechanism is implemented via a sensor system. The sensors collect measurements every 0.01 seconds (corresponds to 100Hz). By evaluating the measurements, the 3B1/3B1=ST knee joint detects the current motion phase of the user.

1.6 Safety Instructions

INFORMATION

Advise your patients of the information in this section.

⚠ CAUTION

Failure to comply with the Safety Instructions. Failure to follow the safety instructions below can lead to control errors or malfunction of the 3B1/3B1=ST knee joint and result in risk of injuries for the patient as well as damage to the 3B1/3B1=ST knee joint.

1.7 Scope of Delivery

- 1 pc. 3B1/3B1=ST knee joint
- 1 pc. Remote control 4X350
- 1 pc. Power supply unit 757L16-*
- 1 pc. Charger 4E60-*

1.8 Recommended accessories

- 1 pc. Rohradapter 2R20/2R21

1.9 General Safety Instructions

⚠ CAUTION

Alignment and adjustment errors. When fitting and adjusting the prosthesis, errors can occur, which can cause malfunctions of the knee joint including loss of function due to structural failure. That can cause the patient to fall.

- Participation in an Otto Bock product training course for the 3B1/3B1=ST knee joint is obligatory before the first fitting. Additional product training courses may become necessary to qualify for product updates.
- Correctly entering the foot size, the prosthesis dimensions and the body weight are important criteria for the quality of the fitting. If the values are too high, the prosthesis may not trigger the swing phase. If the values are too low, the prosthesis may trigger the swing phase at the wrong time.

⚠ CAUTION

Use of unsuitable prosthetic components. If unsuitable prosthetic components are installed in the prosthesis, malfunction of the joint and a loss of functionality due to structural failure may occur. This may subsequently cause the patient to fall.

Combine the 3B1/3B1=ST knee joint with adapters and feet tested by Otto Bock only (see Overview of the Components, page 2).

Otto Bock shall not accept any liability if the 3B1/3B1=ST knee joint is used with prosthetic feet other than those specified.

⚠ CAUTION

Manipulation of system components. Independent changes and / or modifications of system components may lead to malfunction of the knee joint and a loss of functionality due to structural failure, which could subsequently cause the patient to fall.

- Any changes or modifications to the knee joint may limit its use.
- The knee joint may only be opened and repaired by authorised Otto Bock technicians and the battery may only be handled by Otto Bock Service Centres (batteries are not user-serviceable).

⚠ CAUTION

Improper use of the knee joint. Any kind of overloading or excessive strain as well as improper use can lead to defects with subsequent malfunctions of the knee joint and a loss of functionality due to structural failure. This can subsequently cause the patient to fall.

The patient must be instructed in how to handle the 3B1/3B1=ST knee joint properly and be informed of the patient instructions below.

⚠ CAUTION

Improper use of the remote control. Improper use may damage the remote control. This can lead to malfunction of the remote control and result in unexpected actions of the joint. This can subsequently cause the patient to fall.

Instruct the patient in how to use the remote control correctly based on the patient information (646D428=*). Advise the patient of the following patient instructions.

⚠ CAUTION

MyMode switching via remote control. When the remote control is used to switch MyMode, the damping properties of the knee joint are changed. In certain situations (e.g. under bending strain when cycling), this could cause the patient to fall.

Instruct the patient in how to use the MyMode switching in MyModes based on the patient information (646D428=*). Advise the patient of the following patient instructions.

⚠ CAUTION

Transport damage. Mechanical impact or stress during transportation of the knee joint such as shocks and vibrations can lead to:

- Defects and resulting knee joint malfunctions;
- Battery and hydraulic unit defects, resulting in leaks
- Or to loss of functionality due to structural failure.

This can cause the patient to fall as well as result in skin irritation.

When transporting the knee joint, use the packaging in which it was delivered.

⚠ CAUTION

Consequences of product deterioration. Wear and tear on system components can lead to malfunction of the knee joint, which may subsequently cause the patient to fall.

In the interest of patient safety and in order to maintain reliability and protect the warranty, the specified service intervals must be observed.

1.10 Patient Information

⚠ CAUTION

Magnetic interference. The knee joint can malfunction when near high-voltage power lines, transmitters, transformers, CT scanners or other sources of strong electromagnetic radiation (such as security systems for goods in department stores). This can cause the patient to fall.

Avoid proximity to strong sources of magnetic and electric interference (e.g. transformer stations, transmitters).

⚠ CAUTION

Thermal overloading. Extended exposure to high temperatures can lead to defects and result in malfunction of the knee joint with a loss of functionality due to structural failure. This can cause the patient to fall.

Avoid areas with extreme temperatures (see Section 10.5 "Technical Information").

⚠ CAUTION

Mechanical overloading. External mechanical impact or stress such as shocks and vibrations can lead to:

- Short circuits in the electronics and battery, resulting in malfunction of the knee joint;
- Defects of the battery and the hydraulic unit, resulting in leaks;
- Or to loss of functionality due to structural failure.

This can cause the patient to fall as well as result in skin irritation.

Do not expose system components to mechanical vibrations or shocks.

⚠ CAUTION

Penetration of dirt and humidity. Penetration of dirt and humidity into the system components can lead to:

- Short circuits in the electronics and battery, resulting in malfunction of the knee joint;
- Defects in the hydraulic unit, causing leaks;
- Or loss of functionality due to structural failure. This can cause the patient to fall as well as result in skin irritation.
- Ensure that neither solid particles nor liquids can penetrate the system components. Should the knee joint come into contact with liquid, remove the cosmetic cover and let the components dry. The joint must then be sent to an authorised Otto Bock Service Centre for inspection. Please contact your prosthetist.
- If the 3B1/3B1=ST knee joint comes into contact with salt water, immediately clean it with a cloth moistened with freshwater and let it dry. The knee joint must then be sent to an authorised Otto Bock Service Centre for inspection. Please contact your prosthetist.

⚠ CAUTION

Improper use of the joint. Any kind of overloading or excessive strain as well as improper use can lead to:

- Defects and resulting knee joint malfunctions;
- Loss of functionality due to structural failure;
- Or battery and hydraulic unit defects resulting in leaks.

This can cause the patient to fall as well as result in skin irritation.

The 3B1/3B1=ST knee joint was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

Careful handling of the prosthesis and its components not only increases their service life but, above all, ensures your personal safety. Should the prosthesis be subjected to extreme impacts (e.g. by falling or similar), it must be inspected immediately for possible damage by a prosthetist. If necessary, the responsible prosthetist will pass the prosthesis on to Otto Bock Service.

⚠ CAUTION

Overheating of the hydraulic unit. Extended heavy use (e.g. lengthy downhill walks) can lead to:

- Overheating of the hydraulic unit and resulting malfunctions of the knee joint
- Or to defects in the hydraulic unit and associated leaks.

This can cause the patient to fall as well as result in skin irritation. Touching overheated components can also cause burns.

Please take note of pulsating vibration signals. They indicate the risk of overheating. As soon as these pulsating vibrations begin, all activities must be stopped and the hydraulic unit be allowed to cool down. You may resume your activities once the pulsating vibration signals stop.

If activities are continued despite the vibration signals, there is the risk of overheating the hydraulic element and, in extreme cases, of damage to the 3B1/3B1=ST knee joint. The knee joint should be brought to an authorised Otto Bock Service Centre for inspection.

⚠ CAUTION

Danger of falling when descending stairs.

- When walking down stairs, the banister or handrail should always be used and the prosthetic foot should be placed on the step so that the heel (max. centre of the foot) is close to the edge of the step to facilitate rollover.
- If the audible alarm sounds, stop descending immediately and check whether stance phase control is active (see Section 10, Important User Instructions).
- Observe the vibration and audible signals (beeps) of the 3B1/3B1=ST knee joint.
- Particular caution is required when carrying children down stairs.

⚠ CAUTION

Improper MyMode switching. If an improper MyMode switching procedure into basic mode or vice versa occurs, there is a danger of falling (see Section 7.5 MyMode Switching Using the Remote Control)!

- When you switch to a MyMode, check the function, as you may have activated a MyMode other than the intended one (see Section 6 MyMode Motion Switching and Section 7.5 MyMode Switching Using the Remote Control).
- Ensure that you return to basic mode when you have completed your activities in the MyMode.

⚠ CAUTION

MyMode switching via remote control. The patient can use the remote control to initiate different actions changing the damping behaviour of the knee joint. In certain situations, this can cause the patient to fall.

If you inadvertently selected the wrong action with the remote control (vibration or audible signal), take the strain off the 3B1/3B1=ST knee joint and correct the command.

⚠ CAUTION

Consequences of product deterioration. Wear and tear on system components can lead to malfunction of the knee joint, which may subsequently cause the patient to fall.

For your own safety and to maintain operational safety and preserve the warranty, comply with the specified service and inspection intervals.

⚠ WARNING

Danger of accidents when driving a motor vehicle. The ability of leg prosthesis wearers to operate a motor vehicle is determined on a case-by-case basis. Criteria include the type of fitting (amputation height,  lateral or bilateral, residual limb conditions, design of the prosthesis) and the individual abilities of the leg prosthesis wearer. All persons are required to observe their country's national and state driving laws when operating motor vehicles. For insurance purposes, drivers should have their driving ability examined and approved by an authorised test centre. In general, Otto Bock recommends that vehicles should be professionally retrofitted to meet each user's individual needs (e.g. automatic transmission). Risk-free driving must be ensured even when the leg prosthesis is not functioning.

⚠ CAUTION

Malfunction of the knee joint. Malfunctions of the knee joint can cause you to fall.

Observe the vibration signals and audible warnings (beep signals) of the 3B1/3B1=ST knee joint.

INFORMATION

When using exoprosthetic knee joints, servomotor, hydraulic, pneumatic or brake load dependent control functions can cause movement noise. This kind of noise is normal and inevitable. Usually, it does not cause any problems.

If movement noise increases noticeably during the life cycle of the the knee joint, the knee joint should be inspected by a prosthetist immediately.

⚠ CAUTION

Non-active safety mode. If the safety mode can no longer be activated, there is the risk that the patient will fall.

If the 3B1/3B1=ST knee joint can no longer be set to safety mode (e.g. in the event of a short circuit due to water entry), under certain circumstances the patient must actively secure the 3B1/3B1=ST knee joint with the muscles in their residual limb on stepping on their heel until they reach the prosthetist, or until the prosthetic limb has been replaced, and go to the prosthetist's practice without delay.

⚠ CAUTION

Danger of pinching in the hydraulic unit. Avoid reaching into the knee joint mechanism, as this results in a risk of pinching.

2 Installation of X-Soft and connection to the Bluetooth® environment

2.1 Minimum hardware requirements for PC with 32-bit operating systems

- PC with a Pentium® IV 650 / 3.4 GHz processor (x86) or faster
- 1024 MB RAM (memory)

2.2 Minimum PC hardware requirements for PCs with 64-bit operating systems

- PC with an Intel Core2 (x64) processor or faster
- 2048 MB RAM (memory)

2.3 Recommended PC hardware specifications for 32-bit and 64-bit operating systems

- At least 1 GB of free hard drive space
- Graphics card with Open GL support, minimum resolution of 1024 x 768, 32 bit or 16.7 million colours
- 1 available USB port for
- Otto Bock USB 2.0 Bluetooth® Adapter (e.g. 60X5 Bionic Link PC)
- CD-ROM drive or DVD-ROM drive
- Pointing device (e.g. mouse) and keyboard
- Monitor, minimum resolution 1024x768
- Internet connection

2.4 Software requirements

2.4.1 Supported operating systems

The following 32-bit operating systems are supported:

- Microsoft® Windows XP Professional SP2 or higher
- Microsoft® Windows XP Home Edition SP2 or higher
- Microsoft® Vista (all editions except Microsoft® Vista Starter)
- Microsoft® Windows 7

The following 64-bit operating systems are supported:

- Microsoft® Windows 7

INFORMATION

Please note that you require the "USB drivers 2.0" driver version for Bluetooth® connection via 60X5 BionicLink PC under Windows 7.

2.4.2 Additional required software components

The following additional software components are required and installed during installation of the software (if they are not installed on the PC already):

- Microsoft® XML 6.1
- Microsoft® NET Framework 3.5

INFORMATION

Before launching the install, the installation program checks to see if you have sufficient free space on your hard drive and returns an error message if this is not the case. If this occurs, a corresponding amount of space must be freed up on the hard drive. All components required by the Otto Bock software will be copied onto your hard drive automatically by the installation program.

2.5 Installing

To start the installation program, launch Microsoft® Windows and insert the X-Soft CD into the CD-ROM drive. You must have administrator rights to install the software.

After starting the setup program, the Setup Wizard will take you through a number of screens where you can enter the following information:

- Read and confirm the licence agreement
- Enter the name and company

1) Enter the product code provided in the CD cover, and click the "Next" button.

2) Enter the company information. Fields marked with an asterisk (*) are mandatory.

- 3) You will then receive a summary of your data that will be transmitted to Otto Bock during the registration process.
 - To request a license number, click the “E-mail registration” button. Your data is sent to Otto Bock per e-mail.
 - If you do not have an e-mail account but do have access to the internet, click the “Copy to clipboard” button. Send the clipboard contents to registration.austria@ottobock.com using a webmail service (e.g. T-Online, Yahoo, etc.).
 - Alternatively, click the "Print" button in order to print a hard copy and fax it to Otto Bock.

Please send this fax to: +43 -1 56 79 85

Otto Bock Healthcare Products GmbH - Kaiserstraße 39 - 1070 Wien - Austria


- 4) Click the “Cancel” button in order to interrupt the installation process and continue it after you receive the license number.
- 5) Your personal license number will be sent to you within a few working days (within one hour for e-mail requests).

- 6) Restart the installation process in order to complete the registration by entering the license number into the corresponding field (for example, see: e-mail "Austria, Registration", last point "Type in License: xxxxxxxx"). Click the "Next" button.

- 7) The path and authentication method for the patient database are selected in the next step. If the database is installed locally, accept the default settings and click the “Next” button.
- 8) Select the “Install” button to start the installation process. The installation can take a few minutes.
- 9) The next window informs you that the installation is complete.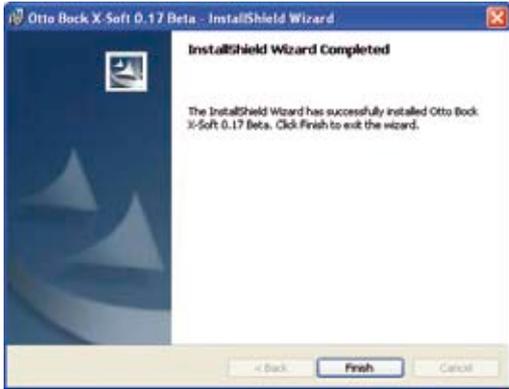

10) Click the “Finish” button. The installation process is concluded. An icon to launch the Otto Bock Data Station has been added to the Windows desktop.

Pentium is a registered trademark of Intel Corporation.

Microsoft and Windows are registered trademarks of Microsoft® Corporation.

X-Soft cannot be accessed until the Otto Bock Data Station has been launched.

INFORMATION

The Otto Bock Data Station is a superordinate software platform used to launch the corresponding software programs. Therefore, the installation program does not create a separate icon for X-Soft.

2.6 Uninstalling

Use the Windows uninstall function to uninstall X-Soft. Open the “Control Panel” and select “Add or Remove Programs” to display a list of all installed software packages. Select the “X-Soft” entry. When you click the “Add/Remove” button, all components of X-Soft are deleted from your hard drive. However, your patient data and the Data Station will be retained.

3 Alignment and adjustment of the prosthesis

3.1 Charging the 3B1/3B1=ST knee joint

INFORMATION

Advise your patients of the information in this section.

INFORMATION

The battery can only be charged at temperatures above 0°C.

It takes approx. 6-8 h for the battery in the knee joint to charge fully. The better the connection between the charger and the receiver on the knee joint (e.g. not interrupted by the stocking), the faster the charging process.

1. Connect the power cord to the charger with correct polarity (Figure 1).
2. Start the inductive charging process by attaching the charger to the receiver at the rear of the knee joint. The charger is held in place by a magnet (Fig. 1 a/2b). Check the charging status via the LEDs at the side (Fig. 3). The charging process can be monitored via the colour of the illuminated ring (Fig. 2b) (see the table below).

3. Remove the charger from the receiver to terminate the charging process. The 3B1/3B1=ST knee joint performs a self-test and is then ready for operation, provided it is connected to the AXON Tube Adapter. This is indicated via a single vibration and beep signal. If this is not the case, an error has occurred. If this occurs please send the knee joint to the prosthetist or authorised Otto Bock Service Centre.

INFORMATION

Hold the joint still while it is performing the self-test, i.e. immediately after removing the charger. Otherwise, error messages may occur. They can be rectified by attaching and removing the charger.

INFORMATION

Charging is no longer possible when the distance between the transmitter and receiver is greater than 2mm. However, charging should not be a problem if there is a sock or Superskin between the transmitter and receiver of the charger. (See also the charging process table).

Charging status indicator on the battery charger:

LEDs	0	1	2	3	4	5
Charging status	0%-10%	10%-30%	30%-50%	50%-70%	70%-90%	>90%

Charging process table:

Status	Status LED	Charging status indicator
The power supply is connected, but the charger is not attached to the knee joint.	 Continuous green	All five LEDs off
Charger connected to knee joint, good connection	 Purple pulsing (4-second cycle); ends after 1 minute	Charging status indicator (green) according to the number of LEDs lit
Charger connected to knee joint, bad connection	 Weak purple, ends after 1 minute	Charging status indicator (green) according to the number of LEDs lit
a) Charger overheating b) Knee joint overheating/too cold c) Connection may be improved by reducing the distance between the charger and receiver.	 Continuous yellow	a) 2nd and 4th LED illuminated a) 1st, 3rd and 5th LED illuminated c) 3rd LED illuminated (See Section 3.1, Fig.3)
Charger fault (error may be rectified by disconnecting and reconnecting the power supply)	 Flashing red	LEDs indicate an error code.

The charging status of the knee joint is also shown on the display of the remote control, and signaled via vibration of the joint (see table below):

Charging status display on the remote control:

Remote Control Display	Vibration Signal of the Knee Joint	Charging status	Remaining operating time
Battery symbol empty	3x vibration	25	approx. 24 h
Battery symbol flashing	5x vibration	10	approx. 6 h
No connection	10x vibration and beeping, then deactivation	5	0h deactivation

INFORMATION

When the knee joint is fully charged (all 5 LEDs at the side light up (Fig. 3)), the maximum use period of the knee joint before the next charging process is approx. 4 days, depending on patient activity. However, we recommend charging the knee joint overnight every day.

3.2 Connecting the AXON Tube Adapter

1. Determine the length of 2R20/2R21 AXON Tube Adapter required with the configuration software X-Soft.
2. Shorten the AXON Tube Adapter to the determined value with the 719R3 AXON Tube Adapter cutter (Fig. 1). Smooth the cutting surface with a 718S2 countersink and sandpaper (Fig. 2). If necessary, smooth by machine the material raised at the outer edge when shortening the AXON Tube Adapter, and always countersink the interior to prevent damage to the AXON Tube Adapter cable.

INFORMATION

Do not cut the AXON Tube Adapter below the 365 mm value on the tube adapter scale.

CAUTION

Risk of cable damage. If the cable is damaged when cutting the AXON Tube Adapter to length, malfunctions of the knee joint may follow. This can subsequently cause the patient to fall.
Ensure the cable is not damaged.

INFORMATION

If the AXON Tube Adapter is disconnected when the knee joint is activated and the charger is not attached, a safety mode is activated and the knee joint sends a warning signal. If the AXON Tube Adapter is connected again and then the knee joint is reset by attaching and removing the charger, it returns to normal mode.

3. Attach the foot to the AXON Tube Adapter. To do so, tighten the screws on the AXON Tube Adapter slightly with an Allen key (size 4 mm) (approx. 4Nm). Fit the AXON Tube Adapter to the foot such that the printed scale faces the front (Fig. 3).

4. Attach the cable of the AXON Tube Adapter to the cable of the 3B1/3B1=ST knee joint (Fig. 4).
5. Push the protruding cable loop into the AXON Tube Adapter (Fig. 5). When the AXON Tube Adapter has been shortened to the minimum length, the plug must be inserted in the cavity (Fig. 6). The cable loop must then be stored carefully.
6. Push the AXON Tube Adapter 60 mm into the 3B1/3B1=ST knee joint (Fig. 7). Corrections to min. 40 mm and max. 70 mm insertion depth of the AXON Tube Adapter into the knee joint are permitted.

CAUTION

Inserting the AXON Tube Adapter. If the AXON Tube Adapter is not inserted far enough into the knee joint, functional losses due to structural failure can result. This can subsequently cause the patient to fall.

For reliable operation, the AXON Tube Adapter must be inserted at least 40 mm.

INFORMATION

If the length must be adjusted by pushing the AXON Tube Adapter in or out during subsequent trials, the patient must always be seated. Before the prosthetic limb is loaded with bodyweight after adjusting the length, the screws on the tube clamp of the knee joint must first be tightened to the prescribed torques.

- Turn the foot outwards slightly, and tighten both screws on the terminal clamp slightly using an Allen key (size 4 mm) (approx. 4Nm) (Fig. 8).

INFORMATION


If

- The AXON Tube Adapter is shortened excessively
 - The foot is replaced with one with a lower system height
 - If a shorter AXON Tube Adapter is used in the event of a service joint
- intermediate adapters (e.g. 4R84) are used to compensate for the height difference.
If you do so, the knee axis-ankle adapter distance must be re-entered in the software.
In general, the AXON Tube Adapter should be left as long as possible.

3.3 Starting the Program


There are three ways to start X-Soft:

- "Start" command in the File menu
- Click  in the Windows system menu
- Key combination Alt +F4.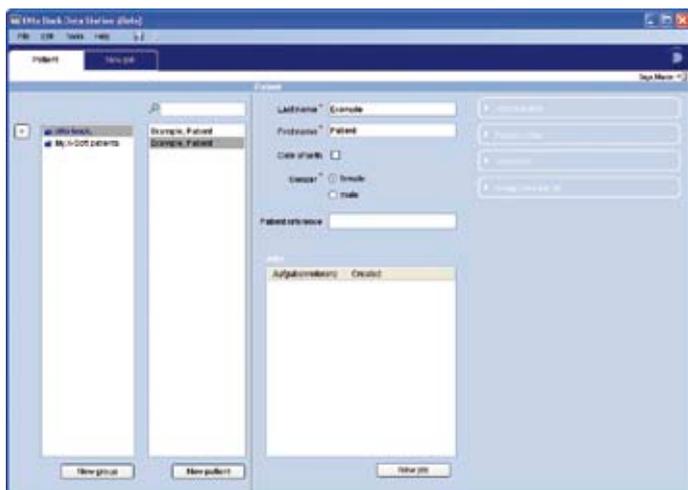

- To start X-Soft click the "New task" tab.
- To start X-Soft, click the "X-Soft" software in the "New task" tab.

Entering the password for the first time

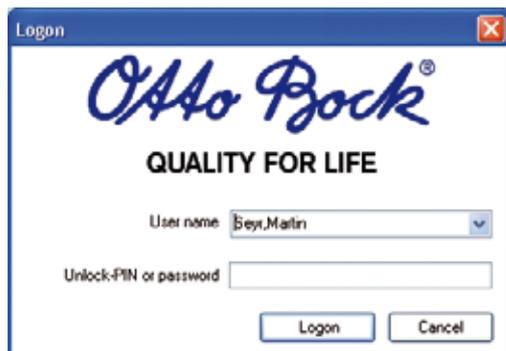The password entry prompt is displayed after you start X-Soft. When entering the password for the first time, you must enter the Unlock-PIN, which was distributed as part of training on the 3B1/3B1=ST knee joint.

After entering the Unlock-PIN, you will be asked to enter and confirm a personal password (chosen by you).

The Unlock-PIN distributed as part of training on the 3B1/3B1=ST knee joint can always be used instead of the personal password. Upon doing so, you will once again be asked to enter and confirm a personal password (chosen by you).

3.4 Closing the Program


There are three ways of closing X-Soft:

- "Exit" command in the "File" menu
- Click  in the Windows system menu
- Key combination Alt + F4.


3.5 Establish connection between the 3B1/3B1=ST knee joint and X-Soft

INFORMATION

Ensure that BionicLink PC 60X5 is installed and connected to the PC used.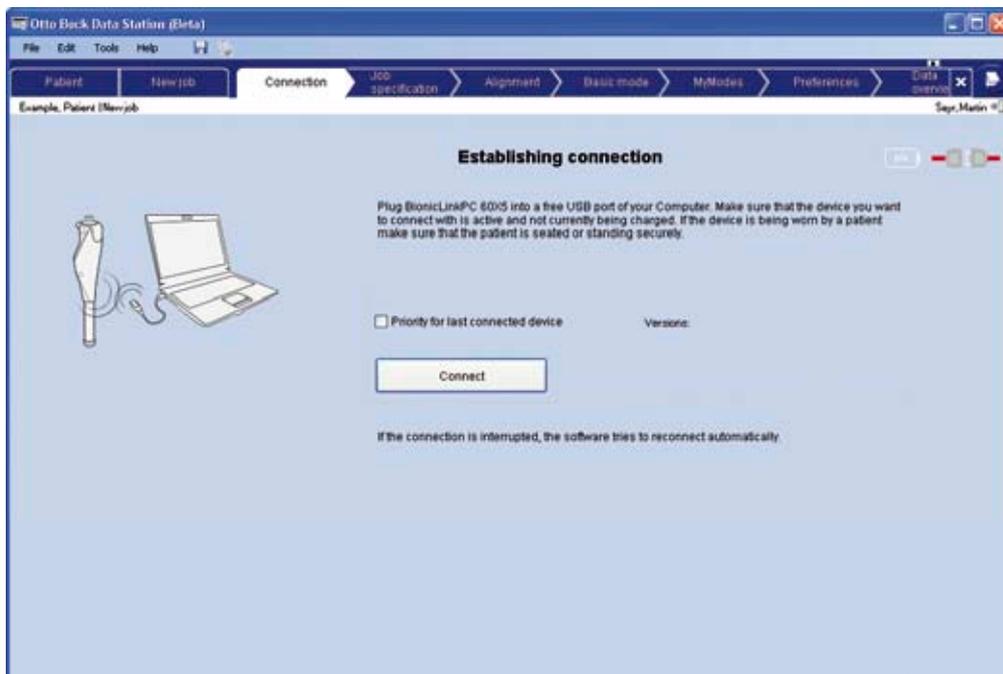

1. Click the **Connection** tab to establish a Bluetooth® connection between X-Soft and the 3B1/3B1=ST knee joint. To select the last knee joint connected, select the **Prioritise last knee joint used** checkbox.
2. Click the **Connect** button to establish the Bluetooth® connection between X-Soft and the 3B1/3B1=ST knee joint. If multiple 3B1/3B1=ST knee joints are in range, the serial numbers are shown in a pop-up window. Click to select the required 3B1/3B1=ST knee joint. Click the **OK** button in the pop-up window to establish the Bluetooth connection between X-Soft and the selected 3B1/3B1=ST knee joint.

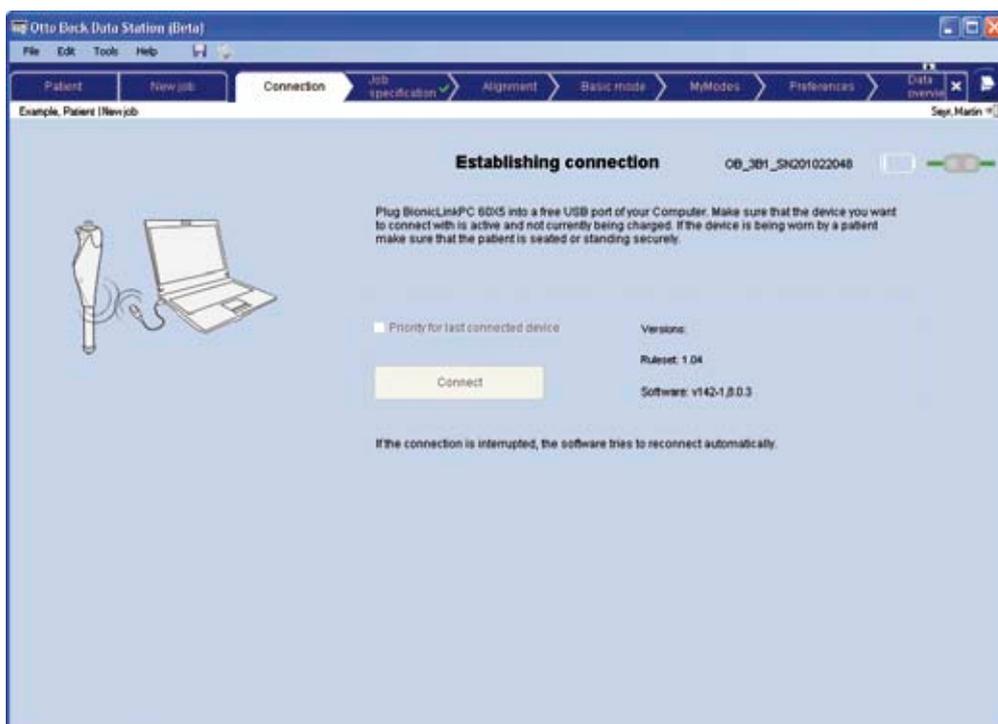If a connection exists, then the name of the knee joint is displayed and the connection icon is displayed at the top of the window in green.

The knee joint emits brief beep and vibration signals to indicate to the patient and prosthetist that settings can now be made.

After a connection has been successfully established, the data is imported from the knee joint and copied to the corresponding fields in the Otto Bock Data Station on the P C.

INFORMATION

The range of the Bluetooth® connection is around 10 metres, but this can vary depending on the local conditions. The prosthesis should not be removed from range while a connection is established.

INFORMATION

If the knee joint is in a safety mode (charger attached, AXON Tube Adapter disconnected or due to a malfunction), certain functions cannot be performed during the adjustment process (knee joint is blocked for structural setup, switching to the MyModes).

If the knee is in a safety mode, this will be displayed alternatingly with the serial number in the top right corner.


3.6 Entering the basic data in X-Soft

Entering the basic data in the **Task details** window provides the basis for calculations of the motion control and individual setup recommendations for the prosthetic limb.

CAUTION

Incorrect data entry in X-Soft. If incorrect data is entered in X-Soft, this can adversely affect the behaviour of the prosthetic limb. This may subsequently cause the patient to fall.

Always enter the weight in the unit set. Ensure that the two dimensions required are selected and entered correctly.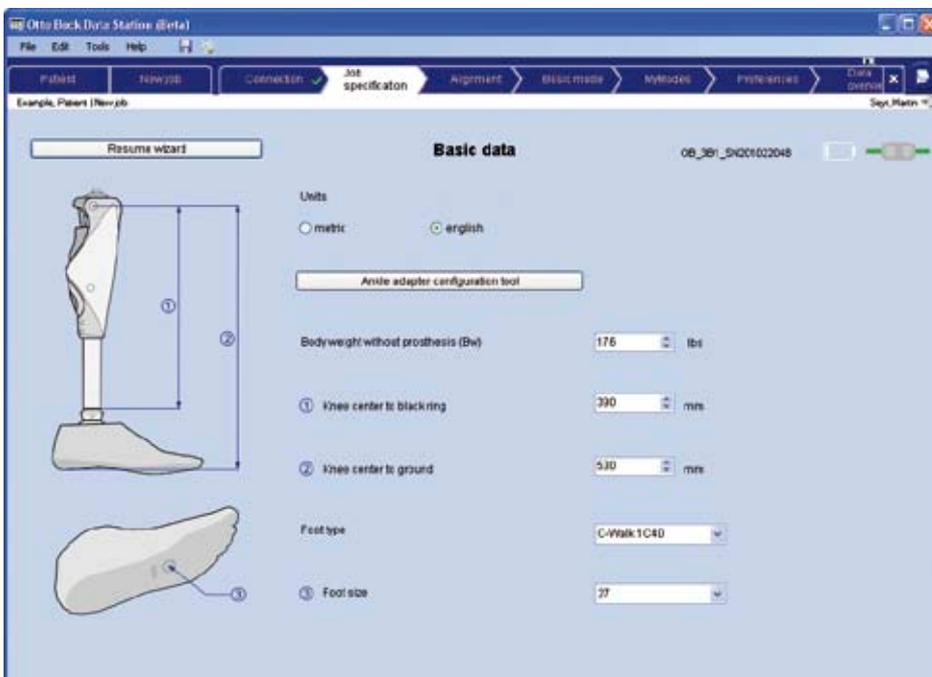

Ankle adapter configuration tool

Use the button to summon the ankle adapter configuration tool.

1. Select the foot type.
2. Select the foot size. Only available sizes for the selected foot type are displayed.
3. The system height of the foot is preselected from a data table. If additional adapters have been used, enter the system height of the foot manually (for example, when using a 4R84, 32mm have to be added to the system height of the foot).
4. Enter the preliminary estimated distance from knee-center to ground (see also information box below). Corrections can of course be made later when the patient tries on the prosthesis. This number only serves to determine the required length of the ankle adapter.
5. Select the ankle adapter type (2R21 with or ~~2R21~~ without torsion unit).
6. Read from the display, at which marking the ankle adapter must be cut (cutting length) and to which marking the ankle adapter must be inserted into the knee frame (insertion length).

INFORMATION


The selection of the foot type is not acknowledged by the knee joint via a vibration or beep signal, as this information is only used in X-Soft.

INFORMATION

Generally, the distance from the knee axis to the ground is equal to the distance from the medial tibial plateau to the ground + 20 mm of the patient's sound leg. If the residual thigh is very long or for ~~exarticulation~~ of the knee, the distance from the knee axis to the ground must be reduced accordingly. Factors which must be taken into account for this include the thickness of the load cushion at the end of the residual limb and the strength of the socket material. One possible procedure for determining the distance from the knee axis to the ground is to measure the distance between the end of the socket and the ground, while the patient stands upright with horizontally aligned hips and a correctly fitted socket (without a shoe on the contralateral side). 27 mm must then be deducted from this distance. Any imprecision in this method should be compensated via the total tolerance of 30 mm (min. 40 mm - max. 70 mm) in the insertion of the AXON Tube Adapter into the knee joint.

3.7 Entry of the patient's data into X-Soft

Entering the patient data in the **Alignment data** window acts as a basis for calculations for the individual alignment recommendation for basic alignment and for optimising the structural alignment.

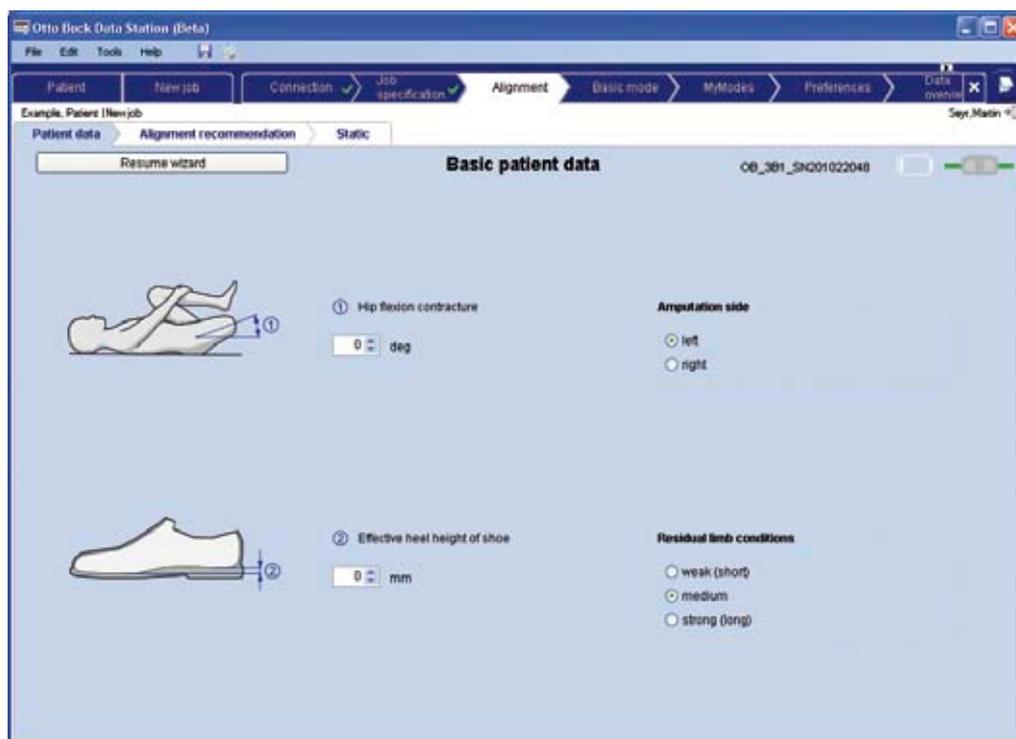Before entering the patient data using the Thomas test (See Section 11 Definition of Terms), determine whether, if any, residual limb/hip flexor contracture is present.

1. Enter the value for the residual limb/hip flexor contracture ①.
2. Enter the effective heel height of the shoe ②.
3. Select the amputation side.
4. Select the residual limb conditions. The emphasis here is on the patient's residual limb strength.

INFORMATION

Before completing entry of the patient data, check that it is complete and correct.


Use the Save function of the Otto Bock Data Station to allow you to load a patient's data again subsequently.

4 Torque values of the screw connections

CAUTION

Failure to comply with torque values. Failure to comply with the torque values of the screw connections can lead to a loss of functionality due to structural failure. This can cause the patient to fall.

Compliance with the specified torque values is essential in order to ensure operational reliability.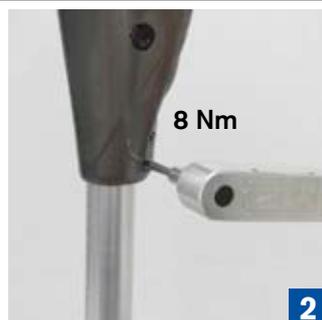


Use a 710D4 torque wrench with a 710Y2=4 hex bit to tighten the screws alternately, in several steps increasing the torque slowly, until the prescribed torque is reached:

1. Tube adapter: 15 Nm
2. Clamp: 8 Nm
3. **Fitting for long residual limb**
Lamination anchor with threaded connector: 15 Nm
Fitting for short residual limb
Rotation adapter or sliding adapter: 10 Nm

5 Basic prosthetic limb alignment with X-Soft

A calibrated alignment recommendation for basic prosthesis alignment is provided to reflect the individual conditions of the prosthesis and patient represented by the patient data entered. All alignment tools use the same reference system (positive or negative values for the distance specifications):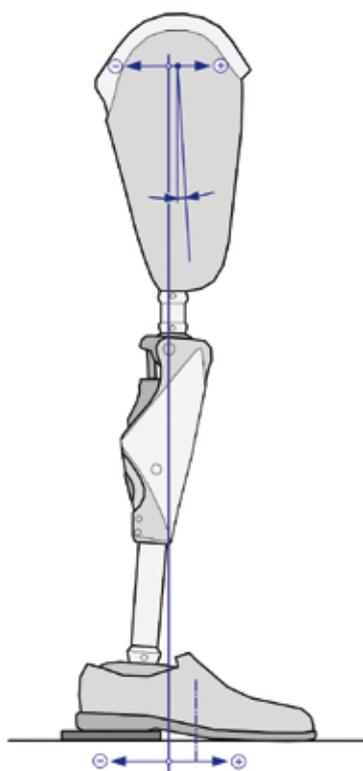


INFORMATION

During basic prosthesis alignment, ensure the knee joint is fully extended. To do so, push the shaft once into the fully extended position.

Selecting the alignment tool:

In the Alignment recommendation tab, select the **alignment recommendation L.A.S.A.R. / PRO.S.A. Assembly or Plumb line / laser line.**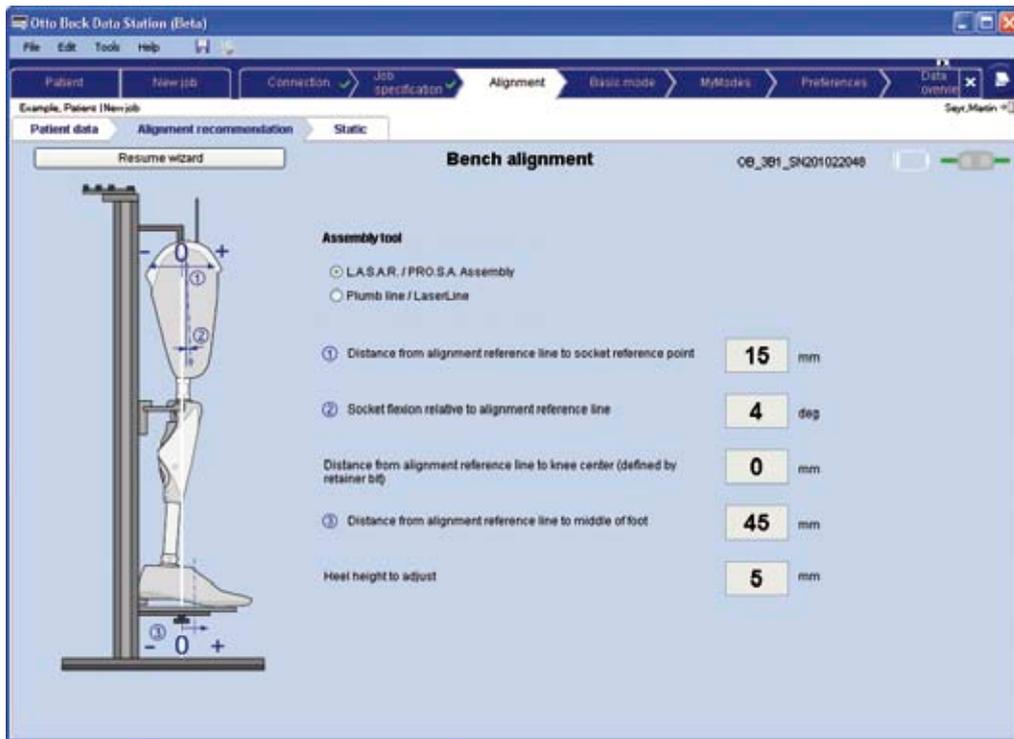

- To determine the socket flexion, indicate the middle of the socket laterally using a proximal and distal point (e.g. on adhesive tape). Draw a line through both points from the edge of the socket to the end.
- The position of the pyramid connection on the socket adapter relative to the knee joint may require the use of a sliding adapter (e.g. 4R112), to reach the target values.

CAUTION

Collision of prosthetic elements with the hydraulic unit. The hydraulic unit can be damaged by collision with other prosthetic elements. A damaged hydraulic unit can lead to malfunctions of the knee joint up to function loss through structural failure. This can lead to the patient falling.

When using shift adapters or similar elements, please make sure that even when the knee is in full flexion no element including the socket collides with the hydraulic unit.


Alignment tool L.A.S.A.R. / PRO.S.A. Assembly:

1. Assemble the prosthesis components using the specified target values.
2. After basic alignment, place the shoe over the prosthetic foot.

Plumb line / laser line alignment tool:

1. Place the shoe over the prosthetic foot.
2. Assemble the prosthesis components using the specified target values.
3. Use a wooden board or a similar object to raise the heel by the specified amount.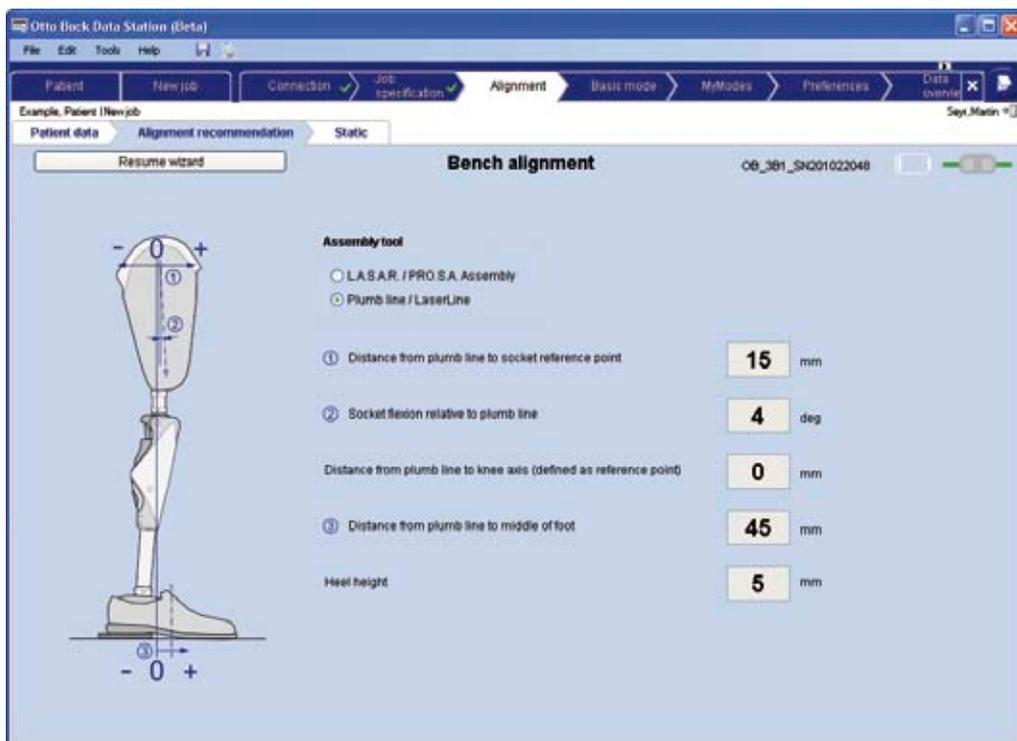

5.1 Static alignment optimisation with X-Soft

The **Static alignment measurements** tab is used to enter specific reference values based on measurement data, which then helps optimise alignment.


A precondition is that the alignment recommendations were followed during basic alignment of the prosthesis.

The objective of optimised alignment is the least possible compensating activity in the residual limb. This is required to support the stance phase and to initiate the swing phase.

Optimised alignment of the prosthesis components can reduce the effort required from the patient.

INFORMATION

During static alignment optimisation, the 3B1/3B1=ST knee joint is automatically locked in the bending direction (see Section 5.2.1 "Adjusting stance phase control"). This should enable the patient to stand in a stable stance that is not prejudiced by the alignment. Patients can only walk with a fully outstretched leg in this situation!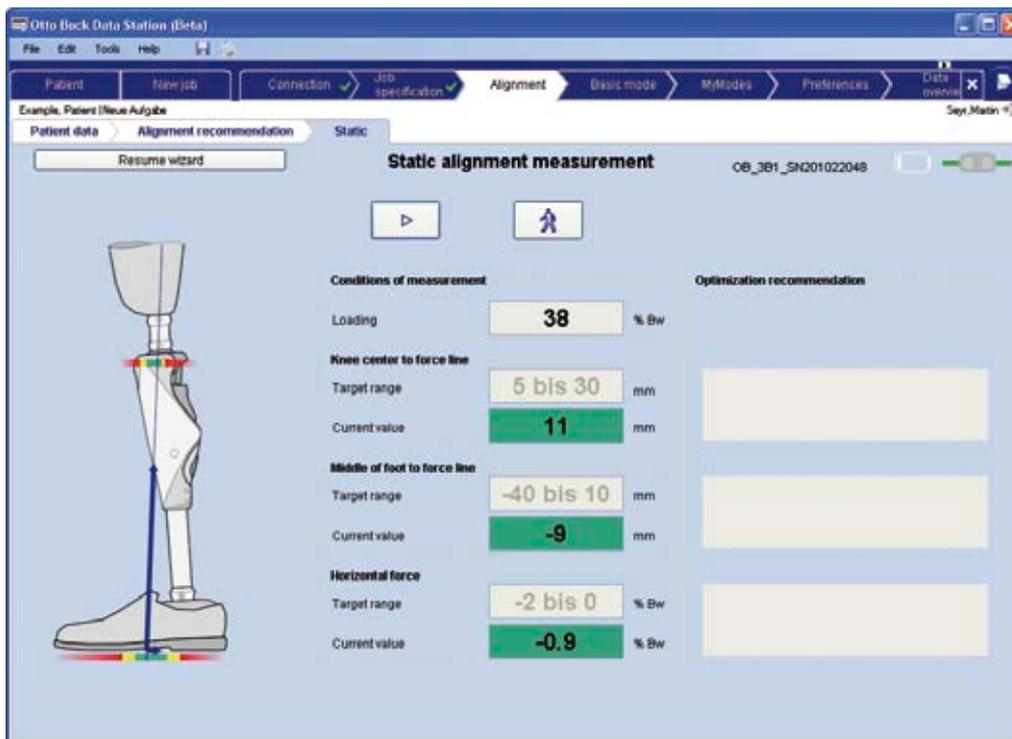

When you select the **Static** window (static alignment measurement), continuous measurement data display starts automatically:


- The almost-vertical arrow represents the ground reaction force.
- The horizontal arrow represents the horizontally applied proportion of the ground reaction force.
 1. Read exact measurements from the numerical displays.
 2.  Select the **Play/ Pause** button to stop (or start) continuous measurement.
 3. This will “freeze” the displayed figures and graphical representation. For each measurement, proposed values are provided in an information window which should lead to the required range.
 4.  Selecting the **Unlock / Lock** button deactivates or activates the knee joint lock. A test run with swing phase control can be performed.
 5.  Select the **Unlock / Lock** button to lock the knee joint for an additional measurement.
 6.  Continuous measurement can be restarted by selecting the **Play/ Pause** button.

5.1.1 Conditions for measurement validity

The following conditions must be met for a valid measurement for static alignment optimisation:

1. Check the following visually:

- Patient is standing on a hard, level surface
- Patient is standing in an upright position
- The patient's arms are hanging loosely down at their sides
- The patient is looking straight ahead
- The tips of the feet are level and the feet are positioned symmetrically.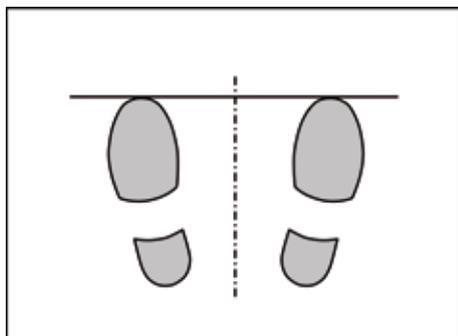


2. Measurements of 3B1/3B1=ST knee joint which are checked in X-Soft:

- The prosthetic leg is loaded with at least 35% of the bodyweight.

5.1.2 Entering measurements

Observe the patient's posture and deviations in the measurements displayed.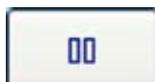

The patient has achieved a stable posture when you can see that the values are deviating only slightly or not at all. Do not freeze the measurement using the Pause button until then.

INFORMATION


If the patient has visibly moved during recording, the saved values are no longer representative and are not suitable for further evaluation. The measurement must be repeated in order to enter suitable values for optimisation proposals.

INFORMATION

Due to the individual habits of the patient and because no information on the alignment in the frontal plane is included in the calculation, the optimisation proposals should be viewed primarily as suggestions that may include inapplicable or even contradictory information in some isolated cases.

Defining +/- sign:

The +/- signs for the distances from the force line are based on the following definition of the + / - signs:

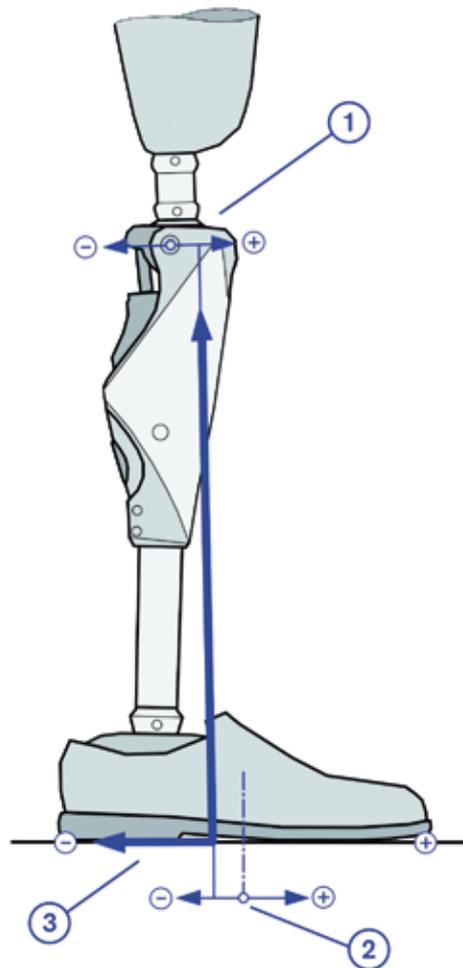① Distance from the knee to the force line:

If the force line is in front of the knee axis (anterior, causes the knee joint to stretch), the "Knee-force line distance" value is positive.

If the force line is behind the knee axis (posterior, causes the knee joint to flex), the "Knee-force line distance" value is negative.

② Distance from foot midpoint to force line:

If the force line is in front of the middle of the foot (anterior, towards the tip of the foot) the "middle of foot - force line" distance value is positive.

If the force line is behind the middle of the foot (posterior, towards the AXON Tube Adapter), the "middle of foot - force line" distance value is negative. The value can be negative even though the force line is still anterior to the adapter.

③ Horizontal force:

The horizontal force is defined to be positive if the foot exerts a backward force on the ground. This may be imagined as the patient trying to pull the foot backwards.

The horizontal force is defined to be negative if the foot exerts a forward force on the ground. This may be imagined as the patient trying to push the foot forward.

5.2 Configuring the prosthesis control via X-Soft

INFORMATION

Adjustments cannot be made while walking. During configuration on the PC, the patient must be standing securely or sitting.

INFORMATION

When the patient becomes accustomed to the new prosthesis, his/her gait pattern will change. It is therefore advisable to perform the complete configuration process again approximately two weeks after initial fitting with the 3B1/3B1=ST knee joint.

5.2.1 Adjusting stance phase control

INFORMATION

On delivery, stance phase control is set for the requirements of an average patient. Thus, the damping values are sufficiently high to guarantee the safety of the patient.

Stance phase control is configured using two parameters: stance phase flexion resistance and stand phase extension resistance.

Stance phase flexion resistance:

The stance phase flexion resistance is the resistance against flexion of the knee which is required for descending stairs or steps or for supported sitting motions.

CAUTION

Danger of falling when descending stairs.

- When walking down stairs, the patient should always use the banister or handrail and the prosthetic foot should be placed on the step so that the heel (max. centre of the foot) is close to the edge of the step to facilitate rollover.
- When the audible signal beeps, the patient must stop walking down the stairs immediately. A test must be run to determine whether stance phase control is active (see Section 10 "Important Information for Users").
- Heed the vibration signals and audible warning signals (beeps) of the 3B1/3B1=ST knee joint.
- Particular caution is required when carrying children down stairs.

Setting stance phase flexion resistance - Ramps and walking down stairs:

Based on bodyweight and lower leg length. In general, no deviation or very little deviation is required from this value. To check this, the patient should sit down on a chair supporting themselves with their hands on the armrests. They lower themselves onto the prosthetic limb and feel the securing effect of the stance phase flexion resistance. Also, the patient should walk down a ramp and then stairs, while holding the banister for safety. If the resistance is subjectively too low or too high, it can be adjusted accordingly.

Adjusting the stance phase extension resistance:

INFORMATION

If the OPG function (see Section 5.2.3) is activated by checking the corresponding box, both the stance phase and the swing phase extension are adjusted automatically. This means that no settings must be made in these areas and the corresponding slider bars in the Swing phase and Stance phase tabs are deactivated.

The stance phase extension resistance is the resistance against knee extension after a stance phase flexion. In a natural gait, shortly after the heel makes contact with the ground while the knee is extended, the leg bends slightly, followed immediately by another extension as the weight is transferred to the front of the foot. This series of movements is called "stance phase flexion". Stance phase extension resistance influences this harmonic and natural gait pattern that cushions the rest of the musculoskeletal system. Stance phase extension resistance determines how smoothly the leg returns to an extended position after stance phase flexion. Set the slide control of the stance phase extension resistance during multiple walking tests such that on one hand, no unpleasant snap extension occurs, and on the other that at a variety of walking speeds, the patient is not given the impression that the knee joint sticks in a bent position.

5.2.2 Adjusting the swing phase control

Swing phase control influences the amount of time it takes for the prosthetic leg to swing through to an extended position, as well as the ground clearance during the swing. Swing phase control therefore influences the symmetry of the gait pattern.

Swing phase angle:

The swing phase angle at which the knee starts to extend again is approx. 65° in a natural gait, regardless of the speed. Depending on the amputation height and individual gait patterns, the patient may require a different value. A very long residual limb (for knee exarticulation) generally requires a greater swing phase angle to achieve sufficient ground clearance when swinging through. The swing phase angle also affects the duration until the prosthetic leg is extended again. The greater the swing phase angle, the later the leg is extended.

Swing phase extension resistance:

The swing phase extension resistance, the resistance against the extension of the prosthetic leg in the swing phase affects the hardness of the limit at which the prosthesis is stopped at full extension. The value also influences the duration of the swing phase. The higher the swing phase extension resistance, the later the leg is fully extended.

INFORMATION

If the OPG function is activated by marking the corresponding checkbox, both the stance phase and the swing phase extension are adjusted automatically. This means that no settings must be made in these areas and the corresponding slider bars in the Swing phase and Stance phase tabs are deactivated.

Adjusting the swing phase flexion angle and the swing phase extension resistance:

Use the slide control to adjust the swing phase angle (starting at 65°), so that the tip of the foot has enough ground clearance when swinging through. Adjust the swing phase extension resistance so that the swing does not end with an uncomfortably hard impact, but not so much that the patient feels that they must wait for the prosthesis to reach its extended position. If the patient has the feeling that the prosthetic limb does not extend in time even at a very low swing phase extension resistance, forcing the patient into an uneven step length, the swing phase angle can be reduced as long as the ground clearance is sufficient. During the adjustment process, have the patient walk several times at noticeably different speeds.

5.2.3 Functions in basic mode

Auto Stance function

The Auto Stance function automatically recognises any situation that puts strain on the prosthesis in the flexion direction but where flexion is not permitted. Examples where this is the case include standing on uneven or sloping surfaces. The knee joint is always locked in the flexion direction when the prosthetic leg is not fully extended, under some amount of load and at rest. When taking weight off the leg or rolling forwards or back, the resistance is reduced evenly to the stance phase resistance. ~~The knee joint is not locked if the above conditions are met but the patient takes a seated position (e.g. when driving).~~

Enabling the Auto Stance function:

Enable the Auto Stance function by selecting the corresponding checkbox if the customer wants to use the function.

Sitting function

If the patient is in a sitting position for more than five seconds, i.e., the thigh is close to horizontal and the leg is not loaded, the 3B1/3B1=ST knee joint switches the resistance to a minimum in both the flexion and extension directions.

INFORMATION

While sitting, the knee joint switches to an energy-saving mode. This energy saving mode is activated regardless of whether the sitting function is enabled or not.

Enabling the sitting function:

Select the corresponding checkbox to enable the sitting function if the patient wants to use this function.

Stairs and obstacles function

Alternating stair ascent

Although the 3B1/3B1=ST knee joint is a passive knee joint, i.e. cannot perform any active motions of its own accord, alternating stair ascent is possible using the 3B1/3B1=ST knee joint. To use this function you must first place the prosthetic foot on the next step up with a bent knee. Then the hip is extended, which also extends the knee. This requires a sufficiently secure hold in the socket and a certain level of residual limb strength. When climbing stairs naturally, knee flexion is achieved by actively raising the lower leg. With the 3B1/3B1=ST knee joint, knee flexion can be achieved via the mass inertia of the lower leg. When raising the extended leg from the ground, extend the hip briefly and then flex it suddenly. The whip action bends the knee. The 3B1/3B1=ST knee joint detects this series of motions and minimises the flexion resistance. Without releasing and/or with stance phase resistance it would only be possible to bend the knee slightly and the patient may feel uncomfortable pressure on the residual limb. This sequence of movements is very similar to the natural sequence if the tip of the foot starts below the next highest step. Here, too, the heel has to be pulled back first. Have the patient practice the motion "Tip of foot under the step" both with the preserved side and with the prosthesis, then the sequence of motions will be intuitive and easy to learn.

When the knee flexion is sufficient, the 3B1/3B1=ST knee joint raises the extension resistance sufficiently so that the patient has enough time to position the foot on the next step before the knee joint is extended again. The supporting area for the foot on the step has to be sufficient so that the heel does not extend back too far over the edge. With too little support area, the lower leg extends too early, placing the patient in a dorsal position. In this phase, the 3B1/3B1=ST knee joint has already maximised the flexion resistance (blocked). The knee joint cannot be flexed further, but only be extended. That ensures that the patient's leg does not buckle if the hip strength is not sufficient for the extending motion.

The patient should support himself/herself with their hand on the contra-lateral side. A smooth wall is also sufficient for this purpose. This lateral support is intended to prevent the residual limb from twisting in the socket. Twisting can lead to unpleasant surface tension between the skin and the socket. Lateral support also improves balance.

When the knee joint is fully extended, the initial position has been reached. The patient can then climb the next step or continue walking normally.

Overcoming obstacles

The stairs and obstacles function can also be used to overcome obstacles. Instead of achieving the necessary ground clearance of the prosthetic foot via lateral abduction of the extended leg, the described whip motion can be used to do so. This is also possible from the walking motion. Take the strain off the prosthetic leg when the knee joint is extended, then perform the whip motion, which switches the 3B1/3B1=ST knee joint into the stairs and obstacles function. This delays the extension movement, giving the patient enough time and ground clearance to step over an obstacle. The prosthetic leg is not fully extended when lowering the heel. Since the knee joint is locked in the flexion direction during this phase, putting weight on it does not pose a hazard. You can then resume walking normally.

Enabling the stairs and obstacles function:

Select the corresponding checkbox to enable the stairs and obstacles function if the patient wants to use this function.

OPG function (optimised physiological gait)

The OPG function consists of multiple individual functions. The knee joint is not fully extended at the end of the swing phase (**pre-flex**). The extension resistance is automatically adjusted so that the knee joint comes to a standstill before it is fully extended. This means that the leg is lowered to the ground slightly bent, as in the natural motion, which makes the stance phase flexion easier, reduces the impact on the musculoskeletal system, reducing the limitation on forward motion.

Flexion resistance is further increased in the further motion sequence during stance phase flexions until it is fully blocked, to prevent uncontrolled buckling.

Extension resistance at the end of the stance phase is automatically adjusted so that the effect of the forefoot extends the knee joint with sufficient speed – but not abruptly – in order to be ready for the next swing phase.

When negotiating ramps, flexion resistance is controlled according to the slope of the ramp. The flatter the ramp, the closer the flexion resistance sequence is to that when walking on a level surface, where temporary locking can occur. The steeper the ramp, the closer flexion resistance is to stance phase resistance.

When walking on a ramp, flexion resistance is reduced to a minimum at the end of the stance phase while shifting the load from the prosthetic leg to the preserved leg (**angle-dependent swing phase**). This means that the knee joint can be flexed without restriction when pulling the prosthetic leg forwards, which increases the ground clearance and makes catching the tip of the foot far less likely.

The improved ground clearance and the automatic adjustment of the flexion resistance enables fluid transfer between two extremes: the gait pattern for walking on the flat, during which the hip and the knee are extended at the end of the stance phase; and the gait pattern for steeper ramps (from around 10% incline), where the hip always remains in a flexed position and the knee is constantly flexed in the stance phase (**Adaptive Yielding control**).

This behaviour is also advantageous when negotiating uneven surfaces. The effort required for the patient to control the prosthesis using hip movements is lower.


Using this function requires that some patients must abandon long-practiced behavioural patterns. Many patients, especially those with many years of experience with mechanical knee joints without stance phase control, have become so accustomed to securing the prosthesis during heel impact by means of hip extension moment that they need to practice in order to use the OPG function and relax their hip musculature during heel impact.

Enabling the OPG function:

Enable the OPG function by selecting the corresponding checkbox if the patient wants to use this function. When this function is activated, both stance phase extension and swing phase extension are adjusted automatically. This means that the configurations in these areas are not necessary and the corresponding slider bars in the **Swing phase** and **Stance phase** tabs are deactivated.

5.2.4 Selecting MyModes

MyModes can be selected in the order of your choice from the drop down menus. The properties for the MyModes can be adjusted for the patient in the corresponding tabs. In addition to changing the MyMode using the remote control, the first two MyModes can be activated via bouncing motions (see Section 6.1 "Switching into MyModes without a remote control"). Select the required MyModes individually and enable the function for switching via bouncing by selecting the respective checkbox **bouncing on forefoot**.

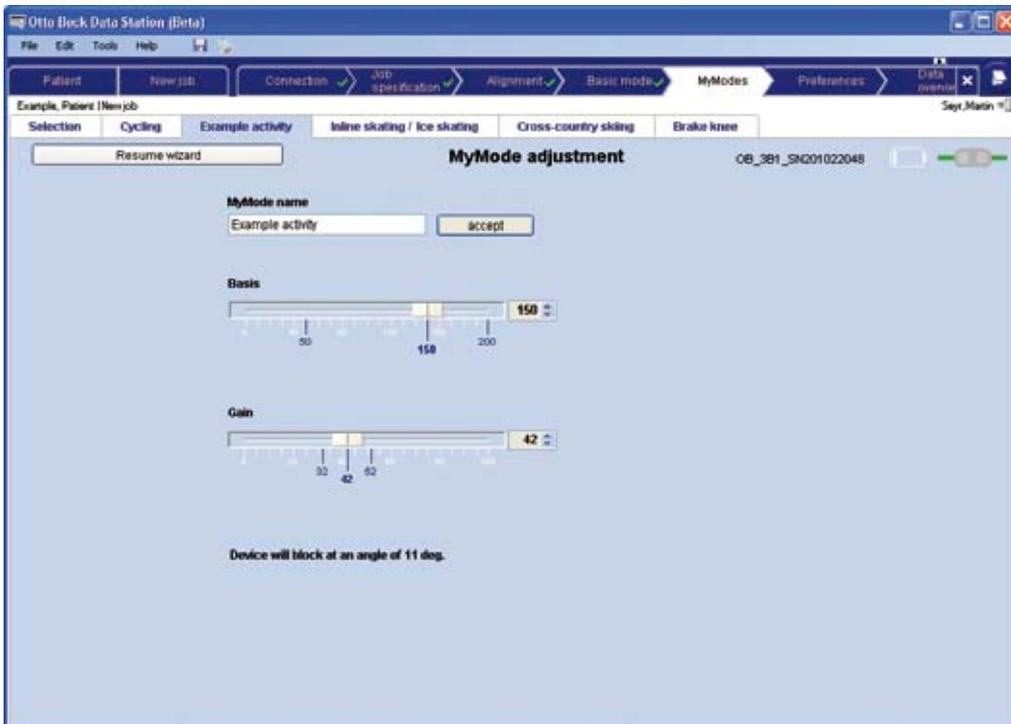The following presets can be selected for the MyModes:

MyMode	Basic	Ascent	Locking angle
Cycling	0	0	∞
Flexed stance	150	42	11°
Inline skating/ice skating	130	35	19°
Cross-country skiing	90	36	29°
Extended stance	200	0	0°
Downhill skiing	180	1	∞
Table tennis	110	28	30°
Tobogganing	100	65	14°
Golf	90	36	29°
User defined	150	0	∞
Brake knee	150	-	Load-dependent

⚠ CAUTION

Improper switching between MyModes. If the switching process between the MyMode and the basic mode and vice versa is not performed correctly, there is a risk of falling (see Section 7.5 "MyMode Switching using the Remote Control")!

- When switching to a MyMode, the function should be tested first, as a different MyMode might mistakenly have been activated (see Section 6 "MyMode Motion Switching" and Section 7.5 "MyMode Switching Using the Remote Control").
- The patient should make sure to return to basic mode when the activities in MyMode have been completed.

Configuring the remote control and selecting the language:

You can use X-Soft to change the user language of the remote control. To do so, select the required language from the drop-down menu. Click the **Configure remote control** button to save the configuration data for the remote control on the prosthesis. This data is required for communication with the remote control.

5.2.5 Setting the resistance in safety mode

If the knee joint software detects a malfunction, or when the battery is flat, the 3B1/3B1=ST knee joint switches to the configurable safety mode (see Section 10.1 "Safety messages and safety modes").

Setting the safety mode flexion resistance:

We recommend a safety mode flexion resistance of 150. It should be configured to enable safe standing without buckling. At a safety mode flexion resistance of 150, normal walking is not possible, but at an insufficient safety mode flexion resistance, the knee joint could buckle, rendering safe standing impossible. On the other hand, some patients with very good control of the prosthesis via the residual limb prefer a safety mode with minimum resistance. This setting is possible.


CAUTION

Danger of falling via a safety mode flexion resistance set too low. If the safety mode flexion resistance is set too low, this can lead to falls!

The safety mode flexion resistance should be configured so that it is possible to stand safely without buckling.

Have the patient walk on a flat surface, on a ramp and stairs with various settings and check their stability.

5.2.6 Setting the pitch and volume of the audible warning and information signals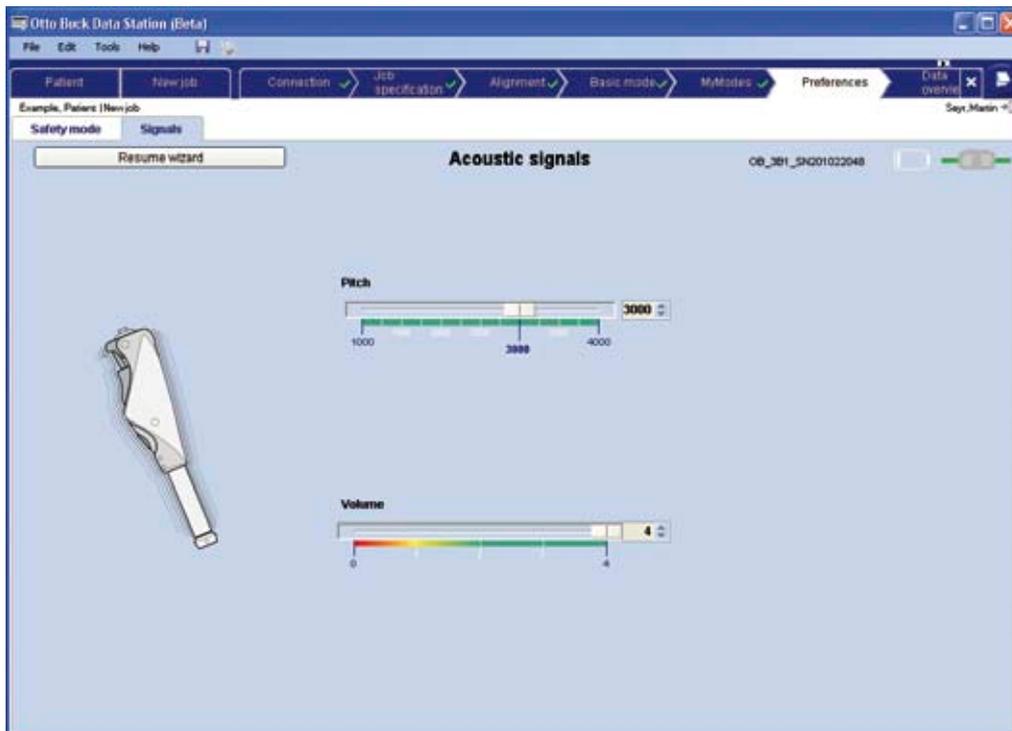

Setting pitch and volume:

Use the controllers for **Pitch** and **Volume** to set the audible signals of the knee joint at values acceptable for the patient. The confirmation signal which sounds after the change corresponds to the setting.

INFORMATION

Warning signals, however, always have the highest volume setting and a fixed frequency.

5.2.7 Data overview after configuring the prosthetic controls


The data overview displays all the latest settings:


All current **values in the knee joint** are shown in the Value in knee joint column.

The **Value in database** column shows the last settings saved in the Ottobock Data Station for the knee joint.

Select the **Expert function** checkbox to change the basic mode settings directly.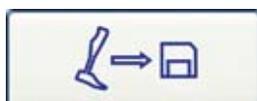

Click this button or the **Save** icon of the Ottobock Data Station (📁) to copy the values from the knee joint to the task.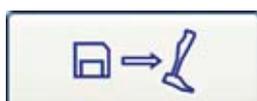

Click this button to overwrite the values in the knee joint with the values from the database.

INFORMATION

To restore the factory default of the knee joint, save the job after connecting to a new knee. By connecting again or using „save as ...“ the job containing the factory default settings can later be used to restore the knee to its original settings.

INFORMATION

Adjustments made by the patient using the remote control are apparent in X-Soft. The basic settings can be readjusted accordingly to give the patient subsequently the original adjustment range around the new setting. Patient adjustments can also be reset or retained.


6 MyMode Motion Switching

INFORMATION

Advise your patients of the information in this section.

6.1 Switching to MyModes without a remote control

If switching by bouncing up and down has been activated in X-Soft, you can switch without a remote control by bouncing up and down on the forefoot three times in a second, with the foot remaining in contact with the ground at all times and the prosthetic leg extended. While doing so, the forefoot has to bear approximately 40% of the total body weight at peak load and less than approximately 25% of the body weight at the lowest load. However, the load may not be taken off the foot fully.

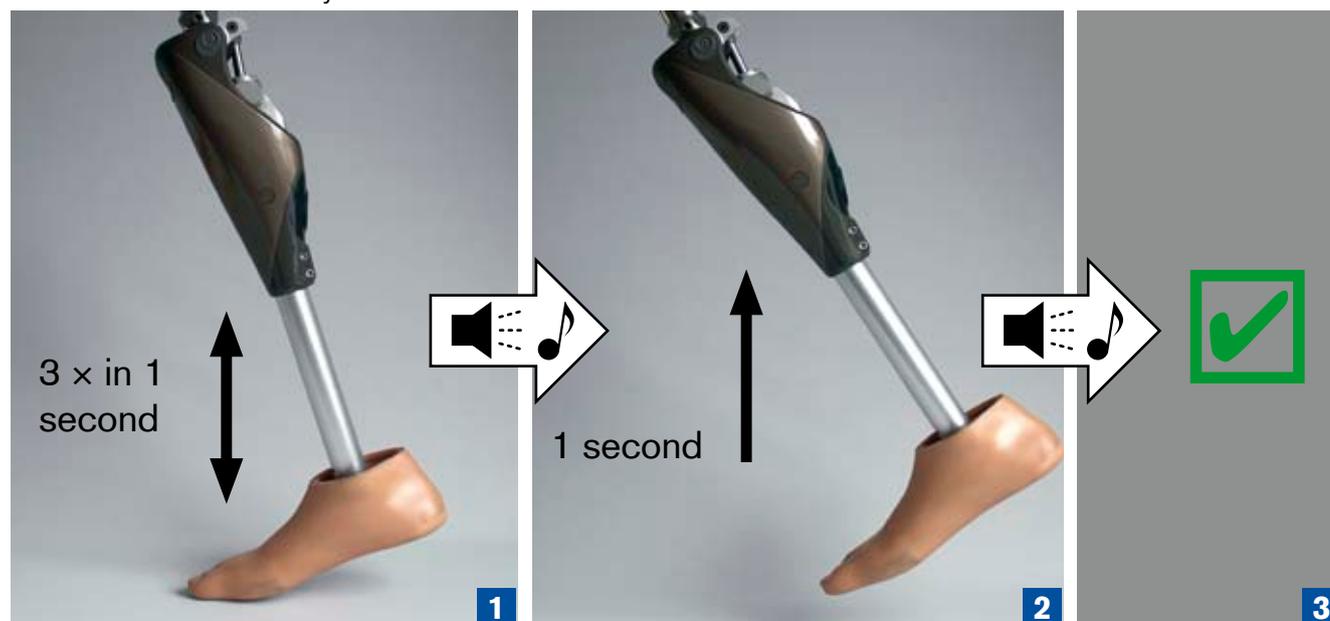After the audible signal sounds (one long beep), your weight must be taken off the prosthesis entirely for at least one second with the knee extended (no ground contact). Then the 3B1/3B1=ST knee joint switches to the first MyMode and emits two short beep signals.

The procedure for switching to the second MyMode is similar, but bouncing up and down four times. In this case, the switch is indicated by three short beeps.

Rock back and forth at least three times to return to basic mode from any MyMode. The switch is confirmed by a short beep.

If only triple bouncing is activated but quadruple bouncing is not, you can access the first MyMode via any number of bouncing motions greater or equal to 3.

INFORMATION

Switching back to basic mode by bouncing up and down three or more times cannot be deactivated. Thus, you can access basic mode even if switching via bouncing is not enabled in X-Soft.

7 Remote control

7.1 Introduction

INFORMATION

Advise your patients of the information in this section.

The 3B1/3B1=ST Knee joint can be used in normal operation in basic mode (incl. optional Auto Stance function, Sitting function, Stairs and Obstacles function and OPG function) and in up to five different MyModes. The MyModes can be activated using the remote control.

Basic mode is used on a day-to-day basis, while the MyModes are pre-programmed for specific movements and / or postures such as bending over a workbench, cycling, inline skating or cross-country skiing.

The configuration software X-Soft can be used to specify how many MyModes are used and what settings are used, but some parameters can also be adjusted with the remote control.

⚠ CAUTION

Improper MyMode switching using the remote control. MyMode switching using the remote control changes the cushioning properties of the knee joint. In certain situations, this can cause the patient to fall.

The patient must stand securely when switching between modes and must carefully confirm whether the desired function was successfully set by the 3B1/3B1=ST knee joint.

⚠ CAUTION

Penetration of water into the remote control. The remote control is not waterproof. If water penetrates the remote control, the device may be damaged (warranty will become void). This can lead to malfunction of the remote control, result in unexpected actions of the knee joint and cause the patient to fall.

The remote control should then be dried at room temperature for at least 1 day. Before using the remote control again, send it to an authorised Otto Bock Service Centre.

⚠ CAUTION

Manipulation on the remote control. Any unauthorised changes or modifications you make to the remote control can lead to malfunction and result in unexpected actions of the knee joint. This can subsequently cause the patient to fall.

Any changes or modifications to the device may limit its usability.

⚠ CAUTION

Unauthorised connection. If an unauthorised connection between a remote control and a knee joint is established, the knee can react unexpectedly. This can lead to the patient falling. Pairing between the knee and the remote control does not require any password or acknowledgment from the knee. It is therefore theoretically possible (with known serial number of the knee or if only one knee is within range of the remote control), that a third party performs pairing with the knee joint and changes settings or switches MyModes.

INFORMATION

The knee joint should consciously be kept still during the switching process.

INFORMATION

For safety reasons, the remote control automatically turns off after two minutes. After the remote control is turned on, approximately three seconds may elapse before it is ready for mode switching processes.

INFORMATION

The range of the remote control is 2-5 m.

7.2 Control elements of the remote control

INFORMATION

Advise your patients of the information in this section.

1. LCD display.
2. Enter button.
3. Navigation buttons: ◀ left, ▶ right, ▲ up, ▼ down.
4. Charging receptacle.

7.3 Initial connection between the remote control and the 3B1/3B1=ST knee joint (Pairing)

1. Press the enter button to activate the remote control. The start-up screen with the date, time and charge status of the remote control is displayed. Without additional input, the remote control turns off automatically after 5 seconds.
2. Press ▼ to attempt to connect the device. The remote control searches for active prostheses (display: "Device scan").
3. When a knee joint is found (serial number is shown), press enter to confirm. If several knee joints are found, the serial number of the desired knee joint has to be selected using the navigation keys (display: serial number). The configuration data from the knee joint is then loaded (display: "Read data").

INFORMATION

This process can take a few minutes.

4. The main menu is displayed.

After successful initial connection with the knee joint, the remote control always connects automatically after activation (press the enter button first, then ▼). No further steps are required.

7.4 Status bar

The status bar shows:

1. Battery status of the remote control
2. Battery status of the knee joint
3. Current MyMode
4. In case of an error in the knee joint (upon service due or when the charger is connected) an exclamation mark appears in the status bar on the right side. In some of these situations, the current MyMode is not displayed.

7.5 MyMode switching via remote control

INFORMATION

Advise your patients of the information in this section.

Once an active connection to the prosthesis has been established, the remote control can be used to switch between MyModes.

1. Select "MyMode" in the main menu and press the Enter key to confirm.
2. A list of the available MyModes is displayed. Press Δ ∇ to select the required MyMode and the enter button to confirm. If you open the MyMode list by accident, you can select the active MyMode again.
3. You can switch off the remote control by selecting and confirming "Remote off".

7.6 Changing prosthesis settings

INFORMATION

Advise your patients of the information in this section.

Once an active connection to the prosthesis has been established, the settings of the respective MyMode control.

1. Select „Device stgs.“ in the main menu and confirm the selection.
2. A list of the available settings for the current MyMode is displayed. Select the desired parameter and confirm.
3. A screen with a scale, the current parameter value and the limit values is displayed. \triangleleft \triangleright Press to change the setting as required and press the enter button to confirm.

4. The list of parameters can be closed without making changes by selecting and confirming the arrow symbol. Press ∇ to access the arrow symbol.
5. Select and confirm "Remote off" to switch off the remote control.

If the settings of another MyMode are to be modified, you must first switch to this MyMode (see Section 7.5 "MyMode Switching Using the Remote Control").

INFORMATION

The prosthetist's setting is in the middle of this scale. After adjustment, you can reset the prosthetist's setting by moving the slider control into the middle.

INFORMATION

The knee joint settings should be optimised by the prosthetist using X-Soft. The remote control is not intended for use by the prosthetist to set the knee joint. The patient can change the behaviour of the prosthesis to a certain extent on a day-to-day basis using the remote control (e.g. while becoming accustomed to the prosthesis). The prosthetist can monitor the changes using X-Soft on the next appointment.

The table below shows the adjustment parameters for the MyModes modes along with a brief description, the adjustment range of the remote control around the value selected by the prosthetist and the absolute value range.

Basic mode:

Slider name in X-Soft	Absolute range	Remote control adjustment range	Description
Stance phase flexion resistance	120-180	±10	Resistance against flexion motion e.g. when walking down stairs or when sitting on a chair.
Swing phase angle	55°-70°	±3°	Maximum flexion angle in the swing phase.
Pitch	1000Hz-4000Hz	1000Hz-4000Hz	Frequency (pitch) of the beeper for confirmation signals
Volume	0-4	0-4	Volume of the beeper for confirmation signals

MyModes except knee braking mode:

Slider name in X-Soft	Absolute range	Remote control adjustment range	Description
Ascent	0-100	±10	Value for speed at which flexion resistance increases as the knee angle increases
Pitch	1000Hz-4000Hz	1000Hz-4000Hz	Frequency (pitch) of the beeper for confirmation signals
Volume	0-4	0-4	Volume of the beeper for confirmation signals

Knee braking mode:

Slider name in X-Soft	Absolute range	Remote control adjustment range	Description
Basic	0-180	±20	Basic resistance for unloaded knee joint
Pitch	1000Hz-4000Hz	1000Hz-4000Hz	Frequency (pitch) of the beeper for confirmation signals
Volume	0-4	0-4	Volume of the beeper for confirmation signals

7.7 Deep sleep mode

INFORMATION

Advise your patients of the information in this section.

The remote control can be used to place the knee joint in a deep sleep mode, in which power consumption is minimised. The knee joint has no function in this status.

It can be woken from deep sleep mode with the remote control or by attaching the charger.

Proceed as follows to activate deep sleep mode:

1. Select "Device stgs." in the main menu and confirm the selection.
2. A list of the parameters available for the current MyMode is displayed. The entry "Deep sleep" is at the end of this list; select it and press the enter button.

3. A confirmation prompt is displayed. Press the enter button again.

Proceed as follows to wake the knee using the remote control:

1. Activate the remote control by pressing the enter button and .
2. Select the "Wake device" menu entry and press the enter button to confirm.
3. The knee joint is reactivated after 20 seconds at the latest with a simple beep and vibration signal. The knee joint is then in basic mode again.

Alternatively, the knee joint can be woken by attaching the charger.

7.8 Querying the prosthesis status

INFORMATION

Advise your patients of the information in this section.

1. Selecting and confirming "Status" displays the following:
 - The total pedometer value ("Total")
 - The daily pedometer value ("Day")
 - The charging status of the prosthesis in percent ("Battery").
 - The estimated remaining operating time of the prosthesis in hours during active use and at rest (e.g. sitting function) ("Stb/Act").
2. Select "Day" and press enter to reset the daily pedometer.

7.9 Remote control adjustments

INFORMATION

Advise your patients of the information in this section.

1. In the main menu, select "Remote stgs." and press enter to confirm.
2. Select the corresponding line to change the date and time. Use the navigation buttons to change the settings.
3. You can then set-up other connections ("Pairing"). Connections can be deleted ("Delete conn.") (see Section 7.10 "Managing prostheses").

4. The font size in the menu bar can be changed by selecting "Zoom".
5. - 6. Press \triangleleft \triangleright on the screen to set the font to normal (1, \triangleleft) or large (2, \triangleright) and then press the enter button to confirm.

7. You can call up information on the firmware version, date, temperature of the remote control and signal strength under "Info". Select the arrow symbol to show the main menu on the display.

7.10 Managing prostheses

Connections with up to four different prostheses can be stored in a single remote control. Conversely, a prosthesis can only be connected to a single remote control. Complete the following steps to add additional connections:

1. In the main menu, Δ ∇ use to select "Remote stgs." and press enter to confirm.
2. Select "Pairing" from the menu that appears.
3. The remote searches for other prostheses in range (display: "Device scan").
4. Press the enter button to confirm if another prosthesis is found. If several additional prostheses are found, the serial number of the desired prosthesis has to be selected using the navigation keys (display: serial number). Press Δ ∇ to select one of the four items and press the enter button to confirm. One of the existing connections can also be overwritten. The configuration files are loaded from the prosthesis and the main menu is displayed.

To delete an existing connection, please follow the steps below:

1. In the main menu, use Δ ∇ to select "Remote stgs." and press enter to confirm.
2. In the menu which is then displayed, select "Delete conn." and press enter to confirm.
3. Use Δ ∇ to select the connection to be deleted and press enter to confirm.

7.11 Connecting to the prosthesis during normal operation

INFORMATION

Advise your patients of the information in this section.

Once the required connections have been configured, the remote control will automatically reactivate the last active connection each time it is turned on.

1. If a connection other than the last active connection is to be selected, select the name of the current connection in the main menu and press enter to confirm.
2. In the list of saved connections which is then displayed, select the required connection and press enter to confirm.
3. If the most recent active connection cannot be re-established, a reduced main menu appears. Choose "Select device" to activate another selected connection.

7.12 Charging the remote control

INFORMATION

Advise your patients of the information in this section.

Pull the charging cable from the charger for the 3B1/3B1=ST knee joint and connect it to the charging socket of the remote control. Regardless of how often the remote control is used, it should last two months on a single charge. The charging status of the remote control is shown at the top left of the display (see Fig. 1, Section 7.3 "Initial connection of the remote control and the 3B1/3B1=ST knee joint (Pairing)"). A full charge of the remote control battery takes 5h.

8 Completing the fitting


⚠ CAUTION

Securing the screws. If the screws are not secured properly, loss of functionality due to structural failure may result. This can subsequently cause the patient to fall.

- The screws must be secured in order to ensure operating reliability.
- Upon finalizing all settings, all screws must be fastened and tightened to the proper torque (see Section 4 "Torque values of the screw connections").
- The screws must then be secured with 636K13 Loctite.

9 Attaching the optional cosmetic covers

If the 3B1/3B1=ST knee joint is fitted with a foam cover, e.g. 3S26, the following steps must be taken with the receiver of the inductive charger (at the rear of the knee joint):

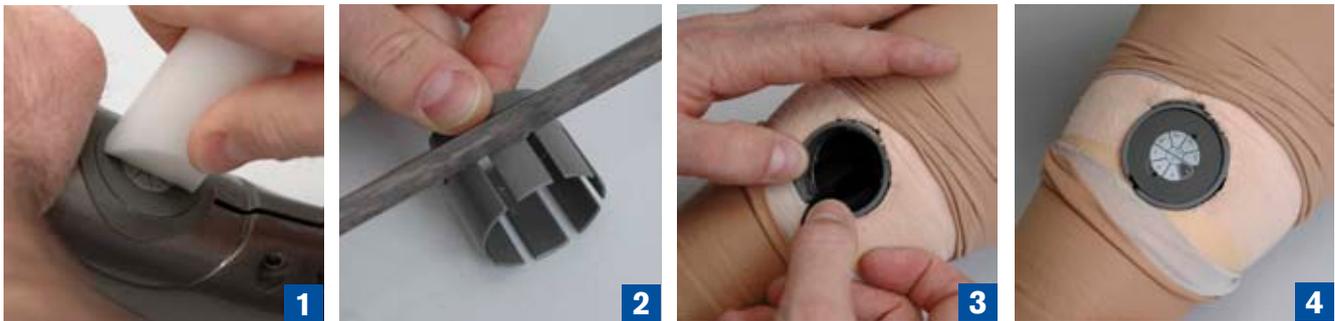1. Remove the receiver of the inductive charger with 4X258 installation tool from the mount in the frame of the knee joint by turning counter-clockwise (Fig.1).
2. Use a suitable tool to shorten 4X259 installation ring on the slotted side so that the length of the installation ring is the same as the material thickness of the foam cover above the circular opening at the rear of the knee joint (Fig.2).
3. Cut a circular opening in the foam cover so that 4X259 installation ring can be attached above the corresponding opening in the frame of the 3B1/3B1=ST knee joint (Fig.3).
4. Glue 4X259 installation ring into the foam cover (Fig.3).

5. Insert the receiver of the inductive charger into 4X259 installation ring using 4X258 installation tool (Fig.4). Ensure that the cable is stowed away securely (Fig.4).
6. The receiver can now be covered by a stocking or similar covering of the foam cover (Fig.5). The magnet holds the charger regardless of this cover (Fig.6).

10 Important information for users

INFORMATION

Advise your patients of the information in this section.

10.1 Safety messages and safety modes

Event	Vibration signal	Beep signal	Required action
Hydraulic system overheated	Every 5s short	-	Reduce activity
Minor error: e.g. Regular service due	After charging, 3 times short	After charging, 3 times short	Walking possible without restrictions, consult prosthetist soon
Medium error E.g. a sensor is not operational	Every minute, 5 times long	5 long beeps on occurrence	Walking possible with restrictions, consult prosthetist immediately
Major error: E.g. Failure of the valve drive	Intermittent continuous vibration	30 long beeps on occurrence	Configurable safety mode; swing phase no longer possible, consult prosthetist immediately
Total failure: No electronic functionality	Continuous vibration	-	Safety mode or unclear condition; swing phase no longer possible; consult prosthetist immediately

In addition to the basic mode and the MyModes, the 3B1/3B1=ST knee joint has a range of safety modes which provide various rest functions depending on the type and severity of the errors. As soon as the 3B1/3B1=ST knee joint detects a fault, at least five long beep signals and 5 slow pulsating vibration signals are emitted. The 3B1/3B1=ST knee joint activates a safety mode and remains in this status until the error is rectified (provided the battery capacity is sufficient).

The OPG function, Stairs and Obstacles function, Auto Stance function and Sitting function are always deactivated in the safety mode; there are also various classes of rest functions:

- Continuous high flexion resistance
- Continuous flexion resistance with the value selected in X-Soft
- Continuous stance phase flexion resistance with the ability to initiate the swing phase.

Depending on the type of error, swing phase initiation lock may or may not be active while walking backwards. You must therefore exercise caution when walking backwards after an error.

The swing phase control and stance phase extension resistance may be available or not depending on the type of error.

If the safety mode was triggered due to a flat battery (see Section 10.2 "Charging status"), charging the prosthesis resets it to basic mode.

There is also an overheating mode to protect the knee joint against excessive temperatures. In this MyMode, stance phase resistance increases as the temperature rises – especially in order to slow down walking downhill so that overheating is not allowed to worsen. Swing phase control is also progressively reduced.

Overheating mode is indicated by brief vibration every 5s.

Basic mode is reactivated once the temperature has fallen sufficiently.

⚠ CAUTION

Safety mode is not active. If the safety mode can no longer be activated, there is the risk that the patient will fall. If the 3B1/3B1=ST knee joint can no longer be set to safety mode (e.g. in the event of a short circuit due to water entry), under certain circumstances the patient must actively secure the 3B1/3B1=ST knee joint with the muscles in their residual limb on stepping on their heel until they reach the prosthetist, or until the prosthetic limb has been replaced, and go to the prosthetist's practice without delay.

⚠ CAUTION

Danger when activating safety mode. The damping characteristics of the joint change when a safety mode is activated. In certain situations, this can cause the patient to fall.

As soon as the sound and vibration signals are emitted simultaneously, the patient must stop all activities involving the leg prosthesis. After approximately 10 seconds, and from a secure standing position, check to see if stance phase resistance is active by slightly bending the 3B1/3B1=ST knee joint repeatedly under controlled weight bearing.

⚠ CAUTION

Safety mode cannot be deactivated. If the knee joint is in safety mode, it is probably defective. When in this condition, malfunctions can occur. This can cause the patient to fall.

If the safety mode cannot be deactivated by recharging the battery, an error has occurred. Contact the prosthetist to have the error corrected.

10.2 Charging status

During normal operation, the charge status of the battery is indicated via vibration signals (in addition to the display on the remote control).

10.3 Storage and bleeding

When storing the knee joint, press the top of the knee to the flexed limit stop. When using the knee joint again, hold the knee joint with the top of the knee facing upwards and push the top of the knee until it is fully extended.

If the 3B1/3B1=ST knee joint is stored in a position other than vertical for an extended period, air can gather in the hydraulic system. This is noticeable through sounds and irregular damping behaviour.

The automatic venting mechanism ensures that all functions of the 3B1/3B1=ST knee joint are fully available after approximately 10 - 20 steps.

10.4 Service interval

The knee joint must be serviced annually to guarantee the functions and ensure the safety of the patient. This service includes the inspection of the sensors and the replacement of worn parts.

You must send the 3B1/3B1=ST knee joint with the installed AXON Tube Adapter (2R20/2R21) and remote control (4X350) and the full charger (4E60-*) incl. power supply unit (757L16-*) for servicing.

10.5 Guarantee

Otto Bock offers an extensive guarantee service according to the current 3B5-X2 Genium guarantee concept (see separate document in delivery package Genium Service Card 646D534). Please comply with the prescribed service intervals (see Section 10.4 Service interval). In your own interest we also recommend you obtain a confirmation from your patient for the provision of instructions and the handing over of the device.

10.6 Technical Information

Operating, storage and transport temperature :	-10 °C to +60 °C
Battery charging temperature:	0 °C to +45 °C
Relative humidity range:	up to 80%
Weight of the knee joint:	1320g
Weight of AXON Tube Adapter 2R20:	290 g
Weight of AXON Tube Adapter 2R21:	530 g
Max. flexion angle:	135°

Minimum clearance (knee center ground):	318 mm
Minimum distal system height with AXON Tube Adapter 2R20:	310 mm
Minimum distal system height with AXON Tube Adapter 2R21:	344 mm
Proximal system height 3B1:	0 mm
Proximal system height 3B1=ST:	27 mm
Operating voltage of charger:	100 V to 240 V AC
Operating frequency of the battery charger:	50 to 60 Hz

10.7 Symbols on the knee joint

Declaration of conformity according to the European Directive for Medical Devices 93/42/EEC and 1999/5/ EU.

This product may not be disposed of with regular domestic waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.

10.8 Symbols on the remote control

Declaration of conformity according to the European Directive for Medical Devices 93/42/EEC and 1999/5/ EC with the number of the specified authority (0681).

This product may not be disposed of with regular domestic waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.

Type BF Applied Part

10.9 Symbols on the inductive charger

Declaration of conformity according to the European Directive for Medical Devices 93/42/EEC.

This product may not be disposed of with regular domestic waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.


Type BF Applied Part

11 Definition of terms

Ground reaction force: Lowering the foot to the ground, and thus applying bodyweight, results in a ground reaction force. The software model indicates the vector of the total effective ground reaction force and the vector of the horizontal component of the ground reaction force.

Terms relating to body planes and position: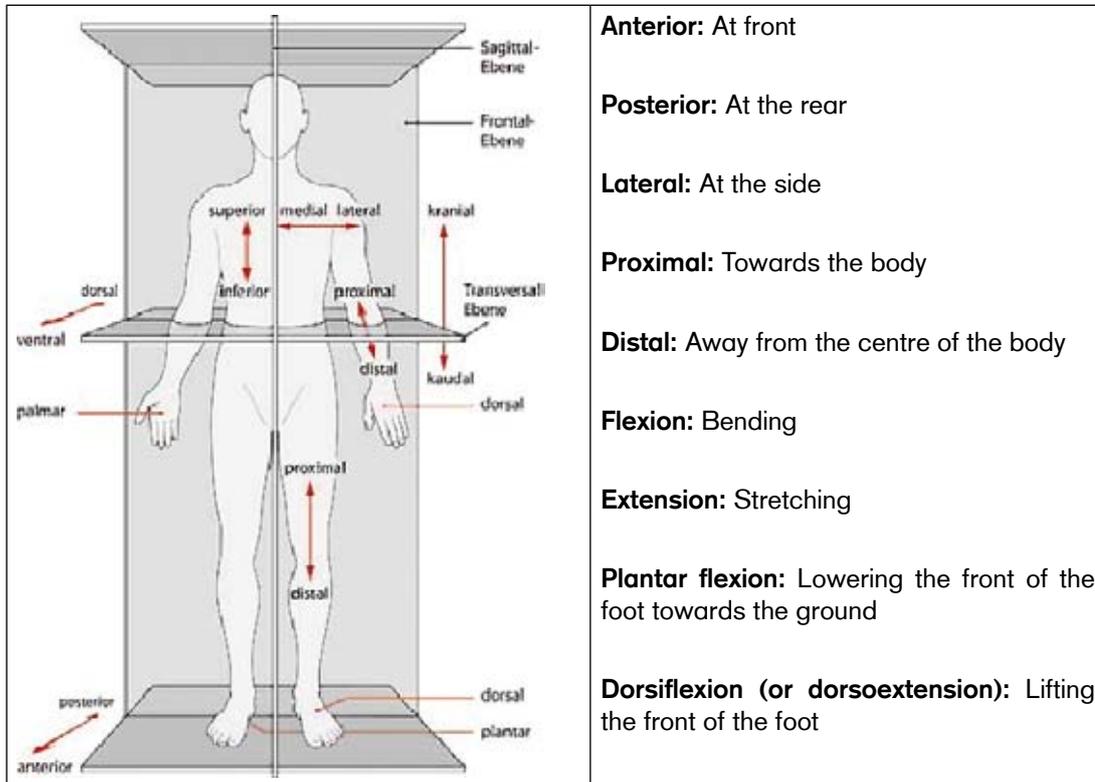


Source: <http://www.spitzer-illustration.com/de/details/086.html>

Contracture: Permanent shortening of muscles, tendons or ligaments in the vicinity of the knee which result in restricted motion.

Abduction: Moving out away from the median plane of the body

Knee center ground measurement: Distance between the center of the 3B1/3B1=ST knee joint and the sole of the prosthetic foot.

Medial tibial plateau-ground measurement: Distance between the upper edge of the calf bone (tibia plateau), which can be felt at the side on the inside of the knee, and the ground. This measurement is to be made when the patient is not wearing shoes.

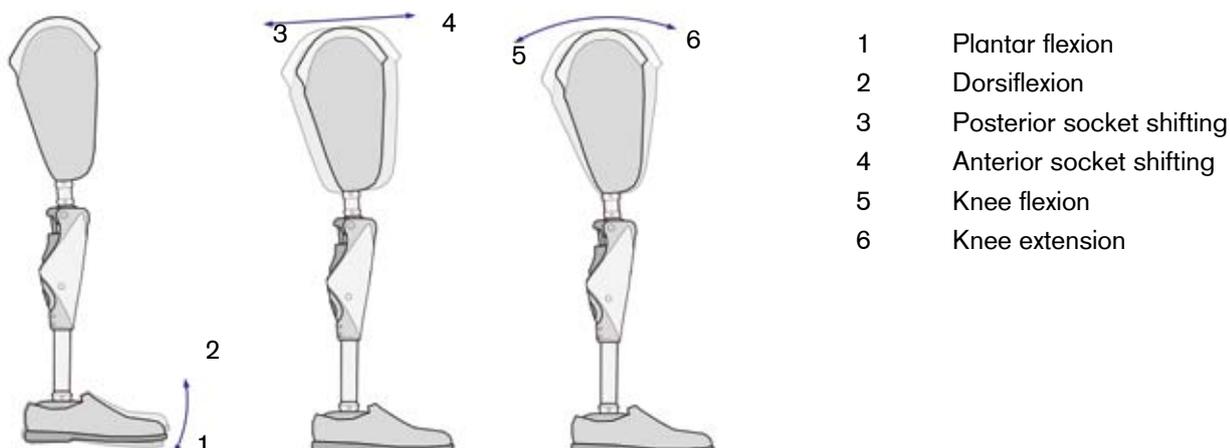
Thomas test:

The Thomas test is a method to determine the range of motion of the hip and, in particular, a hip flexion contracture.

1. To determine the extent of hip flexor contracture, the patient lies on an examination table or other level surface without their prosthesis.
2. Place one hand under the lordosis of the patient's lumbar spine. Now the patient uses both arms to pull the sound leg as close as possible to the body until the spine is pressing on your hand. This fixates the pelvis in the normal position.
3. Determine the hip flexion contracture on the affected side. The hip flexion contracture corresponds to the angle between the thigh and the horizontal plane. If the patient is able to fully extend the hip of the affected side, there is no hip flexion contracture. Enter the value of flexion contracture in X-Soft under "Residual limb / Hip flexor contracture".

Determining a socket reference point:

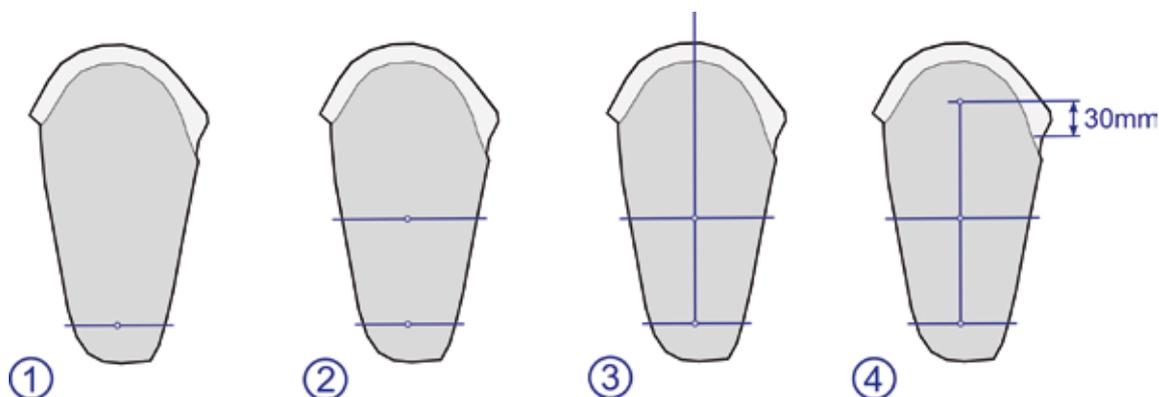1. Mark the centre of the distal part of the socket in the sagittal plane.
2. Mark the proximal centre in the sagittal plane.
3. Connect the two points with a line and extend it to the lowest point of the socket.
4. Establish the tuberosity and define the socket reference point 30 mm above the tuberosity on the connecting line drawn before.

INFORMATION

Since the trochanter is often difficult to recognise as a reference point or because of changes in the hip and pelvic region due to the amputation, this process to determine the socket reference point as the assumed hip rotation point is intended to facilitate prosthesis alignment.

INFORMATION

If L.A.S.A.R./PRO.S.A. Assembly is used, the socket reference point must be marked when the prosthesis is already secured in the alignment unit.

This is due to the 5° outward rotation of the prosthesis in the alignment unit.

12 Legal notes, liability and CE conformity for X-Soft

Disclaimer

The manufacturer explicitly states that this product may only be used in combination with components that were authorised by the manufacturer (see instructions and catalogues). The manufacturer does not assume liability for damage caused by unauthorised component combinations.

License terms for the configuration software X-Soft

Manufacturer: Otto Bock Healthcare Products GmbH

PLEASE READ THIS END USER LICENCE AGREEMENT ("EULA" OR "LICENCE AGREEMENT") CAREFULLY BEFORE YOU INSTALL OR USE THE OTTO BOCK SOFTWARE. BY INSTALLING OR USING THE OTTO BOCK SOFTWARE, YOU ARE AGREEING TO THE TERMS AND CONDITIONS OF THIS EULA.

DO NOT USE THE OTTO BOCK SOFTWARE UNLESS YOU AGREE WITH THE TERMS AND CONDITIONS OF THIS LICENCE AGREEMENT. IF YOU DO NOT AGREE WITH THE TERMS AND CONDITIONS OF THIS LICENCE AGREEMENT, YOU CAN RETURN THE OTTO BOCK SOFTWARE FOR A REFUND OF THE PURCHASE PRICE AT THE LOCATION WHERE YOU ACQUIRED THE SOFTWARE. IF YOU ACQUIRED THE OTTO BOCK SOFTWARE JOINTLY AND IN CONJUNCTION WITH HARDWARE, YOU CAN RETURN BOTH THE HARDWARE AND THE SOFTWARE FOR A REFUND OF THE PURCHASE PRICE.

IF YOU DO NOT AGREE WITH THE TERMS AND CONDITIONS OF THIS EULA, YOU ARE NOT AUTHORISED TO INSTALL OR USE THE OTTO BOCK SOFTWARE.

1. Contractual partner and object of the agreement

- 1.1 By accepting this EULA, you are entering into a contract between yourself and the company Otto Bock; however, if you are acting on behalf of a company when installing or using the Otto Bock software (e.g. as an employee of said company), the contract is concluded between the company and Otto Bock (in which case "you" refers to said company). In case of any doubt, you are a party to the contract.
- 1.2 The license covers the Otto Bock software identified above, including any related documentation (referred to as "Otto Bock software" below). The terms and conditions of this licence agreement also apply to any updates to the Otto Bock software that replace or amend the original software unless such an update is the object of a separate licence agreement. Any possible maintenance of the Otto Bock software or other Otto Bock products is not covered by this agreement; a separate agreement is required.
- 1.3 To the extent that the Otto Bock software will be used to provide recommendations regarding patient fittings (e.g. for certain settings that apply to Otto Bock products after taking customer measurements), please note that such suggestions merely represent non-binding recommendations from Otto Bock. You are still responsible for facilitating a technically correct patient fitting. While Otto Bock will take reasonable steps to ensure the suggestions are correct, it is not responsible for providing technically accurate recommendations.
- 1.4 To the extent that training requirements are specified for the use of the Otto Bock software, the software may only be used by qualified personnel.

2. License

- 2.1 The Otto Bock software is protected by national copyright laws in many countries and by international contracts. Otto Bock prosecutes any violations of the copyrights it holds. You are only granted the right to use the Otto Bock software to the extent required by law or expressly granted to you according to this licence agreement. Otto Bock and / or the licensor of Otto Bock expressly reserve all ownership and / or other rights to the Otto Bock software.
- 2.2 By accepting this EULA, you are granted the non-exclusive right to use the Otto Bock software for your own purposes in accordance with the terms of this agreement. "Own purposes" refers solely to the purposes of the respective contractual partner. In case of orthopaedic supply companies or other companies that provide patient services related to Otto Bock products (e.g. prosthesis adjustments), "own purposes" in terms of this licence agreement are also the purposes of the respective patient who wants to use the Otto Bock products in accordance with their intended use.
- 2.3 The Otto Bock software may only be installed on a single computer unless a supplementary license agreement is concluded. Simultaneous installation on several computers, installation on a network which permits the software to be used on more than one computer at one time, or making the software accessible to the public are all prohibited. Installing the Otto Bock software on a different single computer requires prior removal of the software from the previous computer. In terms of this EULA, Otto Bock products that use the software in accordance with this agreement are also computers.
- 2.4 You do not have the right to modify, edit, translate, rearrange, or otherwise alter the Otto Bock software nor to decompile, disassemble, or reverse engineer the Otto Bock software yourself or via third parties unless expressly permitted to do so by law.

3. Sublicenses, assignment

Issuing sublicenses for the use of the Otto Bock software is prohibited. Renting, leasing, loaning, or otherwise temporarily ceding the Otto Bock software to a third party is also prohibited. The permanent assignment of the usage rights granted to you is only possible when:

- The third party to whom the rights are to be assigned accepts the terms and conditions of this license agreement,
- the entire Otto Bock software, and not just parts thereof, is assigned and
- You do not retain any copies of the Otto Bock software, not even in the main memory or cache of your computer.

4. Warranty

Any warranty for the Otto Bock software is based on the terms and conditions under which you were granted the right to use the Otto Bock software. These normally consist of the applicable Otto Bock General Business Terms and Conditions as well as the applicable legal regulations. This license agreement does not constitute grounds for any additional claims for defects against Otto Bock.

Liability

In case of the culpable breach of so-called cardinal obligations, Otto Bock is liable for the reimbursement of expenses and the payment of compensation (jointly referred to as "compensation" in this Section 5) according to the applicable legal regulations. Cardinal obligations in this sense include all obligations where a breach would compromise the ability to fulfil the purpose of this contract, or where performance is required in order to properly fulfil this contract and where you may rely on regular performance. However, to the extent that breaches of cardinal obligations are due to slight negligence and do not result in death, physical injury, or the impairment of health, your claims to compensation are limited to the amount of the typically foreseeable loss.

Otto Bock is also liable to you according to the provisions of the Product Liability Act; in case of intent or gross negligence; for the loss of life, physical injury, or the impairment of health; where Otto Bock issues any guarantees; and in other cases of mandatory legal liability.

Any other claims for compensation by you against Otto Bock – regardless of the legal grounds and especially due to material defects, defective title, and / or the breach of other obligations arising from the contract on the part of Otto Bock, its legal representatives, employees, or assistants or due to illegal acts – are excluded.

To the extent that the liability of Otto Bock is limited or excluded by the preceding provisions, the same applies to the personal liability of the legal representatives, employees, or assistants of Otto Bock.

The preceding provisions do not affect the burden of proof to your detriment.

Claims for compensation on your part expire within the legal period of limitation, starting at the beginning of said period as established by law.

Applicable law and place of jurisdiction

This licensing agreement is based on the manufacturer's rights. The UN convention on contracts for the international sale of goods does not apply. The exclusive jurisdiction for all disputes arising from or in relation to this licensing agreement is the registered office of the manufacturer.

Severability clause

Should one or more provisions of this agreement be or become ineffective or infeasible in whole or in part, the validity of the remaining provisions of the agreement shall not be affected.

CE Conformity

This device meets the requirements of the 93/42/EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Otto Bock with sole responsibility according to Appendix VII of the guidelines.

13 Liability

The product may only be used under the specified conditions and for the intended purpose, and in conjunction with the modular component combinations tested for the prosthesis, in accordance with the Otto Bock MOBIS® mobility system, and in accordance with the instructions. The manufacturer is not liable for damage caused by component combinations that were not authorised by the manufacturer.

14 CE Conformity

The 3B1/3B1=ST knee joint meets the requirements of the 93/42/EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Otto Bock with sole responsibility according to Appendix VII of the guidelines.

The 3B1/3B1=ST knee joint also meets the requirements of the 1999/5/EC Directive for radio equipment and telecommunications terminal equipment. Conformity assessment was carried out by Otto Bock according to Appendix IV of the directive. A copy of the declaration of conformity can be requested from the manufacturer (see back page).

Service Numbers

USA:	800-328-4058
Canada:	800-665-3327
Great Britain:	1784 744 900

**FCC-Statement:**

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This device may not cause harmful interference, and
- 2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/ TV technician for help.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Caution: Exposure to Radio Frequency Radiation.

This device must not be co-located or operating in conjunction with any other antenna or transmitter.

Responsible party:

Otto Bock Health Care
Two Carlson Parkway North, Suite 100
55447 Minneapolis, Minnesota, USA
Phone + 1-763-553-9464

This device complies with RSS 210 of Industry Canada.

Operation is subject to the following two conditions:

- (1) this device may not cause interference, and
- (2) this device must accept any interference, including interference that may cause undesired operation of this device.

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Caution: Exposure to Radio Frequency Radiation.

The installer of this radio equipment must ensure that the antenna is located or pointed such that it does not emit RF field in excess of Health Canada limits for the general population; consult Safety Code 6, obtainable from Health Canada's website <http://www.hc-sc.gc.ca/rpb>.

Responsible party:

Otto Bock Healthcare Canada Ltd.
5470 Harvester Road
L7L 5N5 Burlington, Ontario
Canada

Manufacturer:

Otto Bock Healthcare Products GmbH

Kaiserstraße 39 | 1070 Wien | Austria

Phone (+43-1) 523 37 86 | Fax (+43-1) 523 22 64

vertrieb.austria@ottobock.com | www.ottobock.at